## Sabouraud CAF Agar + Gentamicin

Selective medium for isolation and cultivation of pathogenic and nonpathogenic fungi.

### DESCRIPTION

Sabouraud CAF Agar + Gentamicin is a selective medium used for the isolation and cultivation of fungi, including dermatophytes, from clinical specimens with mixed bacterial flora.

This medium is also available as gamma-irradiated triple bagged plates, particularly suitable for use in restricted areas like isolators and clean rooms.

### TYPICAL FORMULA

	(g/l)
Enzymatic Digest of Casein	5.0
Enzymatic Digest of Animal Tissue	5.0
Glucose	40.0
Chloramphenicol	0.5
Gentamicin	0.08
Agar	15.0

Final pH 5.6 ± 0.2 at 25°C

### METHOD PRINCIPLE

Enzymatic digests of casein and enzymatic digest of animal tissue provide amino acids, carbon, nitrogen, vitamins and minerals for the growth of fungi. The high glucose concentration along with the acid pH make this medium particularly well suited for cultivating fungi. Chloramphenicol is a broad-spectrum antibiotic inhibitory to a wide range of Gram-negative and Gram-positive bacteria. Gentamicin is an aminoglycoside antibiotic which suppresses the growth of bacteria without affecting fungal growth. Agar is the solidifying agent.

### TEST PROCEDURE

Inoculate the plates by streaking directly the sample onto the agar surface. Streak the specimen as soon as possible after it is received in the laboratory. Incubate aerobically at 20-25°C for 2-7 days.

### INTERPRETING RESULTS

Examine containers for fungal colonies exhibiting typical color and morphology. Biochemical tests and serological procedures should be performed to confirm findings.

Transfer of growth from slants to plated media may be required in order to obtain pure cultures of fungi.

### APPEARANCE

Slightly opalescent, amber.

### STORAGE

Store at 10-25°C away from light. Do not use the product beyond its expiry date on the label or if product shows any evidence of contamination or any sign of deterioration.

### SHELF LIFE

6 months.

**QUALITY CONTROL**

Plates are inoculated with the microbial strains indicated in the QC table.

Inoculum for productivity: 50-100 CFU.

Inoculum for selectivity: 10<sup>4</sup>-10<sup>6</sup> CFU.

Incubation conditions: aerobically, at 20-25°C for up to 5 days (fungi) or at 35 ± 2°C for 48 h (bacteria).

**QC Table.**

Microorganism		Growth
<i>Candida albicans</i>	ATCC® 10231	Good
<i>Trichophyton mentagrophytes</i>	ATCC® 9533	Good
<i>Aspergillus niger</i>	ATCC® 16404	Partially to completely inhibited
<i>Saccharomyces cerevisiae</i>	ATCC® 9763	Inhibited
<i>Escherichia coli</i>	ATCC® 25922	Inhibited
<i>Staphylococcus aureus</i>	ATCC® 25923	Inhibited

**WARNING AND PRECAUTIONS**

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is intended for *In vitro* diagnostic use and must be used only by properly trained operators.

**DISPOSAL OF WASTE**

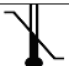
Disposal of waste must be carried out according to national and local regulations in force.

**BIBLIOGRAPHY**

1. Sabouraud. 1892. Ann. Dermatol. Syphil. 3:1061.
2. Larone. 1995. Medically important fungi: a guide to identification, 3<sup>rd</sup> ed. American Society for Microbiology, Washington, D.C.
3. Wehr and Frank (ed.). 2004. Standard methods for the examination of dairy products, 17<sup>th</sup> ed. American Public Health Association, Washington, D.C.

PRESENTATION		Contents	Ref.
Sabouraud CAF Agar + Gentamicin	90 mm ready-to-use plates	20 plates	10235
Sabouraud CAF Agar + Gentamicin	90 mm ready-to-use plates	100 plates	10235*
Sabouraud CAF Agar + Gentamicin	90 mm ready-to-use plates (triple-wrapped and gamma-irradiated)	20 plates	10235S
Sabouraud CAF Agar + Gentamicin	140 mm ready-to-use plates	10 plates	10243

**TABLE OF SYMBOLS**

<b>LOT</b> Batch code	<b>IVD</b> <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
<b>REF</b> Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult Instruction For Use	 Do not reuse

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