RPMI Broth

RPMI Broth double strength

Instructions For Use
ENGLISH

Liquid medium for the determination of broth dilution MICs of antifungal agents for yeasts, moulds and dermatophytes, according to EUCAST.

DESCRIPTION

RPMI Broth is a liquid medium used for the determination of broth dilution minimum inhibitory concentrations (MICs) of antifungal agents for yeasts, conidia forming moulds and dermatophytes, as standardized by the European Committee on Antimicrobial Susceptibility Testing (EUCAST).

TYPICAL FORMULA (g/l)	1x concentration	2x concentration
RPMI-1640	10.4	20.8
MOPS	34.53	69.06
Glucose	18	36
Final pH 7.0 ± 0.1 at 25°C		

METHOD PRINCIPLE

RPMI-1640 is a completely synthetic medium with glutamine and phenol red but without bicarbonate. 3-(N-morpholino) propanesulfonic acid (MOPS) is the buffer agent at a final concentration of 0.165 M. This medium contains 2% glucose (RPMI 2% G) to optimize growth and facilitate the determination of MIC endpoints.

TEST PROCEDURE

Following EUCAST guidelines, the test is performed in flat-bottom well microdilution plate (with a nominal capacity of approximately 300 µl/well). The method is based on the preparation of antifungal agent working solutions in 100 µl volumes per well to which 100 µl inoculum is added. RPMI Broth is used at double strength to allow for a 50% (1:1) dilution, once the fungal inoculum is added. This approach allows the inoculum to be prepared in distilled water.

Refer to appropriate methods outlined in BIBLIOGRAPHY.

INTERPRETING RESULTS

After incubation the presence of turbidity indicates growth of the organism. The lowest concentration of antifungal agent that inhibits the growth of fungi within a defined period of time is the MIC of that organism for that agent. Interpret the MIC by referring to the current EUCAST breakpoints and report the organism as susceptible, intermediate or resistant to the agents that have been tested.

For a complete discussion on EUCAST-AFST, go to <https://eucast.org/astoffungi/>

STORAGE

2-8°C away from light. Do not use the product beyond its expiry date on the label or if product shows any evidence of contamination or any sign of deterioration.

SHELF LIFE

2 years.

QUALITY CONTROL

Appearance of the Medium: Slightly opalescent, amber.

Expected Cultural Response:

Control strain		Inoculum	Incubation	Specification
<i>Candida krusei</i>	ATCC 6258	≤2.5x10 ⁵ CFU/ml	24-48 h 35 ± 2°C	Good growth
<i>Candida parapsilosis</i>	ATCC 22019			
<i>Candida albicans</i>	ATCC 64550			
<i>Aspergillus fumigatus</i>	ATCC 204305			

Please refer to the actual batch related Certificate of Analysis (CoA).

WARNING AND PRECAUTIONS

For *in-vitro* diagnostic use. For professional use only. Operators must be trained and have certain experience in the laboratory methods. Please read the instructions carefully before using this product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.

Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

The product is available in the various configurations listed below. There may be additional product ref. numbers as well. For an updated listing of available products, visit liofilchem.com

Product	Format	Packaging	Ref.
RPMI Broth	Tube	20 x 10 ml	24461
RPMI Broth double strength	Tube	20 x 9.9 ml	24462

This IFU document and the SDS are available from the online Support Center:

liofilchem.com/ifu-sds

RPMI Broth RPMI Broth double strength

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liofilchem.com/ifu-sds

BIBLIOGRAPHY / BIBLIOGRAFIA

1. EUCAST E.DEF 7.3.2 (2020) Method for the determination of broth dilution minimum inhibitory concentrations of antifungal agents for yeasts.
2. EUCAST E.DEF 9.3.2 (2020). Method for the determination of broth dilution minimum inhibitory concentrations of antifungal agents for conidia forming moulds.
3. EUCAST E.DEF 11.0 (2020). How to: perform antifungal susceptibility testing of microconidia-forming dermatophytes following the new reference EUCAST method E.Def 11.0, exemplified by *Trichophyton*.

Above documents are available from <https://www.eucast.org/astoffungi/methodsinantifungalsusceptibilitytesting/>

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