Egg Yolk Emulsion

Instructions For Use

ENGLISH

Liquid supplement for the identification of lecithinase-producing bacteria (*Bacillus* and *Clostridium* species).

DESCRIPTION

A sterile, stabilized egg yolk emulsion (50%) for use in the preparation of various culture media, including the followings:

Polymyxin pyruvate egg yolk mannitol bromothymol blue agar (PEMBA) and Mannitol egg yolk polymixin agar (MYP) for detection of *Bacillus cereus*, according to ISO 7932, ISO 21871 and FDA-BAM;

Tryptose sulfite cycloserine agar (TSC) and Shahidi-Ferguson perfringens agar (SFP) for detection of *Clostridium perfringens*.

TYPICAL COMPOSITION (Per Litre)

Egg Yolk	500 ml
Saline Solution (0.85% NaCl)	500 ml

METHOD PRINCIPLE

When added to a nutrient medium, egg yolk emulsion permits the detection of microbial lecithinase activity.

PREPARATION

Shake the bottle well to suspend any sediment. Aseptically add to the culture medium, sterilised and cooled to 45-50°C. Mix well and pour into sterile Petri dishes.

See details in the Technical Sheet of the specific medium:

- *Bacillus cereus* Agar Base (Mossel), ref. numbers 610114, 620114 and 402710;
- *Bacillus cereus* Agar Base (Pemba), ref. numbers 610136 and 620136;
- *Clostridium perfringens* Agar Base, ref. numbers 610207 and 620207.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical sheet of the medium being prepared.

STORAGE

Store at 2-8°C away from light. Do not use the product beyond its expiry date on the label or if product shows any evidence of contamination or any sign of deterioration.

SHELF LIFE

1 year.

QUALITY CONTROL

Appearance: Yellowish opaque emulsion. May contain a precipitate that can be resuspended.

Microbiological Performance:

Control strains

Bacillus cereus WDCM 00001 (ATCC® 11778, NCTC 10320)

Escherichia coli WDCM 00012 (ATCC® 8739, NCTC 12923)

Bacillus subtilis WDCM 00003 (ATCC® 6633, NCTC 10400)

Please refer to the actual batch related Certificate of Analysis (CoA).

WARNING AND PRECAUTIONS

For In Vitro Diagnostic use. For professional use only. Operators must be trained and have certain experience in the laboratory methods. Please read the instructions carefully before using this product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.

Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

Product	Format	Packaging	Ref.
Egg Yolk Emulsion	Bottle	4 x 50 ml	80219
Egg Yolk Emulsion	Bottle	6 x 100 ml	80124

This IFU document and the SDS are available from the online Support Center:

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liofilchem.com/ifu-sds

BIBLIOGRAPHY / BIBLIOGRAFIA

1. EN ISO 11133:2014+Amd1:2018+Amd2:2020. Microbiology of food, animal feed and water – Preparation, production, storage and performance testing of culture media.
2. FDA-BAM Chapter 14 (2020): *Bacillus cereus*. Bacteriological Analytical Manual.
3. ISO 21871:2006: Microbiology of food and animal feeding stuffs – Horizontal method for the determination of low numbers of presumptive *Bacillus cereus* – Most probable number technique and detection method.
4. ISO 7932:2004: Microbiology of food and animal feeding stuffs – Horizontal method for the enumeration of presumptive *Bacillus cereus* – Colony-count technique at 30°C.
5. Mossel, D.A.A., Koopman, M.J. and Jongerius, E. (1967): Enumeration of *Bacillus cereus* in foods. Appl. Microbiol. 15: 650-653.
6. Donovan, K.O. (1958): A selective medium for *Bacillus cereus* in milk. J. Appl. Bacteriol. 21: 100-103.

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