A.F. Genital System Reagent

Reagents for indole and oxidase tests for use with A.F. Genital System.

DESCRIPTION

A.F. Genital System Reagent is a kit containing Kovac's Reagent, Oxidase Test Stick and Vaseline Oil necessary for the proper use of A.F. Genital System (ref. 74156), a system for detection, count and susceptibility testing of pathogenic urogenital microorganisms.

KIT CONTENT

- 1 x 10 ml vial of Kovac's Reagent.
- 2 x 50 strips of Oxidase Test Stick in plastic tubes.
- 2 x 50 ml bottles of Vaseline Oil.
- 1 empty dropper bottle.

METHOD PRINCIPLE

Kovac's Reagent is used to carry out the indole test into the well 17-PRO.

Oxidase Test is used with the liquid taken from the well 22-NES in order to confirm *Neisseria gonorrhoeae*.

Vaseline Oil is used for the maintenance of a sufficiently anaerobic environment in all the wells of the system other than 6-TR/YE, 16-ESC, 17-PRO, 18-PSE, 20-STF, 21-STR, 22-NES and 23-STG.

REAGENTS

- Kovac's Reagent: 5% (w/v) p-dimethylaminobenzaldehyde dissolved in a solution of 25% hydrochloridric acid and 75% isobutyl alcohol.
- Oxidase Test Stick: paper strips impregnated with 1% (w/v) aqueous solution of tetramethyl-p-phenylenediamine dihydrochloride formulated with stabilizing agents.
- Vaseline Oil.

TEST PROCEDURE AND RESULTS INTERPRETATION

Refer to the package insert provided with A.F. Genital System.

QUALITY CONTROL FOR THE USER

For each reagent, check positive and negative results by using suitable organisms, according to bibliographic references.

PRECAUTIONS

A.F. Genital System Reagent is classifiable as hazardous under current legislation; it is recommended that the Safety Data Sheet be consulted on its use. The product is intended for *in vitro* diagnostic use only and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store at 2-8°C away from light in its original package, until the expiry date shown on the label. Eliminate if signs of deterioration or contamination are evident.

ELIMINATING USED MATERIAL

After use, used Enterosystem 18R Reagent and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

BIBLIOGRAPHY

1. Murray, Baron, Pfaller, Tenorev and Tenover: Manual of Clinical Microbiology (1995).
2. Bayley and Scott's: Diagnostic Microbiology (1986).
3. Edwin H. Lennette: Manual of Clinical Microbiology (1995).

Product	REF	Contents	IFU
A.F. Genital System Reagent	80258	100 tests	www.liofilchem.com/ifu-sds

TABLE OF SYMBOLS

IVD In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
REF Catalogue number	 Fragile, handle with care	 Use by	 Consult instructions for use	LOT Batch code

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