Device for the evaluation of residual antimicrobial power
in urines, milk and other biological fluids.

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DESCRIPTION

ANTIBIOTIC TEST is a device constituted by particular paper discs impregnated with living spores of *Bacillus subtilis* ATCC® 6633 and with a dry culture medium. This device is indicated for the evaluation of residual antimicrobial power in urines, milk and other biological fluids.

KIT CONTENT

Each kit contains:

- 2 cartridges with 25 discs each, impregnated with living spores of *Bacillus subtilis* ATCC® 6633 and with a dry culture medium;
- 50 plastic tubes.

PRINCIPLE OF THE METHOD

The paper disk contains living spores of *Bacillus subtilis* ATCC® 6633 and a culture medium with triphenyl- tetrazolium- chloride (TTC). Spores and medium are released into solution after the addition of the liquid sample to examine (urine, milk or other biological fluid). Substances endowed with antimicrobial activity, eventually present in the liquid sample, inhibit *Bacillus subtilis* spores development, so that liquid in the tube does not change its colour. In the case of lack of substances with antimicrobial activity, *Bacillus subtilis* spores sprout. Vegetative forms multiply in the culture medium and determine the development of a red colour in the sample for the reduction of TTC to formazan, which is a red compound.

COMPOSITION

Each disk is impregnated with living spores of *Bacillus subtilis* ATCC® 6633 at a concentration about 4×10^5 UFC per disk and with a culture medium containing TTC.

COLLECTION AND STORAGE OF THE SAMPLES

In order to perform the test on urine use the same sample of the urinoculture. For collection of milk or other biological fluids, follow the instructions suggested for culture tests on these samples.

Store the samples at 2-8 °C before use.

TEST PROCEDURE

1. Take the cartridge containing the disks from the refrigerator and allow it to reach room temperature.
2. Take a disk of Antibiotic Test from the cartridge and put it into a tube.
3. Add 0.2 mL of urine, milk or other biological fluid to test.
4. Cap the tube and incubate at 36 ± 1 °C for 24 hours.
5. Observe for development of a red colour after 8-12-24 hours.

INTERPRETATION OF THE RESULTS

The development of a red color in the tube of the sample indicates the absence of antimicrobial substances or their presence at a lower concentration than sensitive limit of the methods.

The absence of a red colour in the tube indicates the presence of antimicrobial substances in the sample at a higher concentration than sensitive limit of the method.

QUALITY CONTROL

Each batch of ANTIBIOTIC TEST is submitted to quality control using urine sample free from antimicrobial substances at unknown concentration but with the addition of antibiotics at the following concentrations:

Ampicillin: 5 mcg/ mL

Cefoxitin: 5 mcg/ mL

LIMITS AND WARNINGS

Results obtained with ANTIBIOTIC TEST are qualitative.

PRECAUTIONS

The product ANTIBIOTIC TEST cannot be classified as hazardous under current legislation, nor requires a Safety Data Sheet to be available.

ANTIBIOTIC TEST is a disposable device to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment, by properly trained personnel, knowing approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store ANTIBIOTIC TEST at 2-8 °C in its original container. Keep away from sources of heat and avoid excessive changes of temperature. In such conditions ANTIBIOTIC TEST maintains its validity until the expiry date indicated on the label. Don't use beyond that date. Eliminate without using if there are signs of deterioration.

DISPOSAL OF USED MATERIAL

After use ANTIBIOTIC TEST and the materials come into contact with the sample have to be decontaminated and disposed of in accordance with the techniques used in laboratory for decontamination and disposal of potentially infected material.

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TABLE OF SYMBOLS

 In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Consult instructions for use	 Batch code

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LIOFILCHEM® S.r.l.