Muco-sol

Liquid reagent for the liquefaction of sputum samples.

INTENDED PURPOSE

Muco-sol is a liquid reagent for the liquefaction of sputum samples before microbiological testing. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

DESCRIPTION

Muco-sol is a reagent containing Dithiothreitol (DTT) as active agent and used for the isolation of pathogenic or saprophytic bacteria and fungi. Dithiothreitol (DTT) is the common name for Cleland's reagent.

KIT CONTENTS

10 glass vials containing Muco-sol solution.

METHOD PRINCIPLE

Dithiothreitol (DTT) is the common name for the Cleland's reagent, which is a reducing agent. Once oxidized, DTT forms a stable six-membered ring with an internal disulfide bond. Sodium chloride and potassium chloride are included to perform the fundamental function of providing the ions necessary for microbial growth in order to maintain adequate osmolarity of the culture medium. Disodium hydrogen phosphate and potassium dihydrogen phosphate are the buffering agents as well as sources of phosphorus and potassium.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: culture media, inoculating loop, quality control organisms, test tubes, sterile physiological solution (saline), Pasteur pipet, sterile distilled water.

REAGENTS

Buffer solution with 1,3% dithiothreitol (DTT).

SPECIMEN

Collect sample in appropriate CE marked leak proof containers and transport to the laboratory. Process each sample using the test procedure.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

1. Collect sample in appropriate CE marked leak proof containers.
2. Aseptically add approximately 5 times the volume of sterile saline and agitate to separate the sputum from adherent saliva. Remove saline with a sterile pipette.
3. Add the contents of 1 vial of Muco-sol solution to a bottle containing 92,5 ml of sterile distilled.
4. Add an equal volume of diluted Muco-sol solution to the washed sputum.
5. Shake well or vortex for 30 seconds.
6. Incubate at 37°C in a water bath (or incubator), shaking frequently, until complete liquefaction.
7. If necessary, repeat two or more time point 2 to 6.
8. Inoculate onto a suitable culture medium.

INTERPRETING RESULTS

Refer to the technical sheet of the culture medium being used.

STORAGE CONDITIONS

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration. Unopened vials are stable for 12 months from date of manufacture. Diluted Muco-sol solution is stable for up to 48 h when stored at 2-8°C.

SHELF LIFE

1 year.

QUALITY CONTROL

A representative sample of all lots of Muco-sol is tested for productivity prior to batch release. The tests are performed comparing the results with samples not exposed to Muco-sol.

Productivity is tested by qualitative technique:

A solution ≤ 10000 CFU/ml for each test organisms is suspended in Muco-sol solution; shaken for 30 seconds and kept 10 minutes at $35 \pm 2^\circ\text{C}$ (shake at the beginning and at the end of incubation period). Inoculate ≤ 1000 CFU/ml of each Muco-sol solution with strain and incubate at $35 \pm 2^\circ\text{C}$ for 16-20h.

- *Streptococcus pneumoniae* ATCC 49619
- *Haemophilus influenzae* ATCC 49766
- *Staphylococcus aureus* ATCC 25923
- *Bacteriodes fragilis* ATCC 25285

Please refer to the actual batch related Certificate of Analysis (CoA).

PERFORMANCE CHARACTERISTICS

Performance testing of Muco-sol was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

Some clinical samples are more resistant to liquefaction (see TEST PROCEDURE).

Invalid results can be caused by poor specimen quality, improper sample collection, improper transportation, improper laboratory processing, or a limitation of the testing technology. The operator should understand the principles of the procedures, including its performance limitations, in advance of operation to avoid potential mistakes.

Due to nutritional variation, some strains may result in poor growth or fail to grow on used medium.

WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics center, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

ORDER INFORMATION

Product	Packaging	Ref.
Muco-sol	10 x 7,5 ml	81058

Revision History

Revision	Release Date	Change Summary
0	2023-06-15	Document creation

This IFU document and the SDS are available from the online Support Center:

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