H₂O₂ / Catalase Reagent Droppers

Reagent for rapid detection of catalase enzyme, in dropper bottles.

INTENDED PURPOSE

Reagent for the execution of catalase test on bacterial isolates. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

DESCRIPTION

H₂O₂ / Catalase Reagent Droppers is a solution of hydrogen peroxide used for differentiation and microbial identification on the basis of the presence of catalase enzyme. The enzyme catalase is present in most aerobic and facultatively anaerobic bacteria, with the major exception of streptococci and enterococci.

KIT CONTENT

- 20 bottles of H₂O₂ / Catalase Reagent Droppers (1 ml)

METHOD PRINCIPLE

Hydrogen peroxide, final product of aerobic metabolism of carbohydrates, is extremely toxic for bacteria. Catalase-positive microorganisms, containing the enzyme catalase, break down hydrogen peroxide into water and oxygen. Catalase reagent shows the presence of enzyme catalase with gas bubbles production.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: inoculating loop, pipettes, test tube, culture media, glass slide/microscope slide, physiological solution (0.85 % saline), quality control organisms.

REAGENTS

- Composition of H₂O₂/ Catalase Reagent Droppers is aqueous solution of hydrogen peroxide (3% approximatively)

SPECIMEN

Collect specimens in sterile containers or with sterile swabs and transport to the laboratory. Process each specimen using procedures appropriate for that sample. This product is recommended for use only with pure cultures.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

Prior to proceed with one of the following methods bring the dropper bottle to a suitable temperature.

Slide Test Method

1. Obtain a pure culture of the organism to be tested.
2. Using an inoculating needle or applicator stick, pick a well-isolated colony and transfer to a glass slide.
3. Add 1 or 2 drops of the H₂O₂ / Catalase Reagent Droppers to the smear.
4. Examine immediately for the rapid production of gas bubbles.

Tube or Agar Plate Method

1. Add a few drops of H₂O₂ / Catalase Reagent Droppers to the surface of an 18 – 24 h agar plate or slant that does not contain blood.
2. Examine immediately and after 5 min for the evolution of bubbles.

Broth Method

1. Express the contents of a H₂O₂ / Catalase Reagent Droppers into a 24 – 48 h broth culture.
2. Examine immediately and after 5 min for the evolution of bubbles.

INTERPRETING RESULTS

Positive catalase reactions are denoted by the production of bubbles that occur after the addition of the reagent to the culture.

STORAGE

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

SHELF LIFE

2 years.

QUALITY CONTROL

Appearance of H₂O₂ / Catalase Reagent Droppers: Limpid and clear solution, free of precipitate.

Control strain		Characteristic reactions
<i>Staphylococcus aureus</i>	WDCM 00034 ATCC® 25923 NCTC 12981	Positive reaction: Formation of bubbles of oxygen
<i>Campylobacter jejuni</i>	WDCM 00005 ATCC® 33291 NCTC 13367	Positive reaction: Formation of bubbles of oxygen
<i>Listeria monocytogenes</i>	WDCM 00021 ATCC® 13932 NCTC 10527	Positive reaction: Formation of bubbles of oxygen
<i>Listeria innocua</i>	WDCM 00017 ATCC® 33090 NCTC 11288	Positive reaction: Formation of bubbles of oxygen
<i>Listeria ivanovii</i>	WDCM 00018 ATCC® 19119 NCTC 11846	Positive reaction: Formation of bubbles of oxygen
<i>Enterococcus faecalis</i>	WDCM 00009 ATCC® 19433 NCTC 775	Negative reaction: No formation of bubbles of oxygen
<i>Enterococcus faecium</i>	WDCM 00177 ATCC® 6057	Negative reaction: No formation of bubbles of oxygen

PERFORMANCE CHARACTERISTICS

Performance testing of H₂O₂ / Catalase Reagent Droppers was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

H₂O₂ / Catalase Reagent Droppers may be used as an aid in the identification of microorganisms. Additional biochemical testing using pure cultures is recommended for complete identification.

Culture media that contain blood are unacceptable for this test due to the presence of catalase in erythrocytes. When picking colonies from blood media, avoid carryover when performing the slide test. If a blood medium is used, a control slide catalase test should be performed where a small loopful of the blood-containing agar is tested with the reagent on the same slide as the organism. If the catalase reaction from the colony is much stronger than that from the agar alone, the test can be considered positive.

Dirty glassware can cause false-positive results. Use properly cleaned tubes and slides. Inoculating needles or loops containing iron may produce false-positive results. Avoid contact of the catalase reagent with iron-containing needles and loops.

Bacteria (e.g., *Lactobacilli*) grown on media with low levels or no glucose may yield confusing reactions from pseudocatalase, a non-iron enzyme. The pseudocatalase reaction can be prevented by the addition of 1% glucose to the medium.

Anaerobic cultures grown on a blood-free medium must be exposed to air for a minimum of 30 min before testing.

WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

ORDER INFORMATION

Product	Packaging	Ref.
H2O2 / Catalase Reagent Droppers	20 x 1 ml	87003

Revision History

Revision	Release Date	Change Summary
0	2023-05-15	Updated layout and content in compliance with IVDR 2017/746, version reset to revision 0

This IFU document and the SDS are available from the online Support Center:

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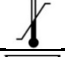
References / Riferimenti

1. EN ISO 11133:2014+Amd1:2018+Amd2:2020. Microbiology of food, animal feed and water – Preparation, production, storage and performance testing of culture media.
2. MacFaddin, J.F. 1980. Biochemical tests for identification of medical bacteria, 2nd ed. Williams & Wilkins, Baltimore MD.
3. Baron, E.J., L.R. Peterson, and S.M. Finegold. 1994. Bailey & Scott's diagnostic microbiology, 9th ed. Mosby-Year Book, Inc., St. Louis.
4. Isenberg, H.D. (ed.). 1994. Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.

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