Aesculin Bile Test

Rapid test for detection of aesculin hydrolysis.

INTENDED PURPOSE

Tests for execution of esculin hydrolysis for identification of Group D streptococci. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

DESCRIPTION

Aesculin Bile Test is used to differentiate Enterococci and Lancefield group D streptococci (*Streptococcus bovis* group) from other streptococci based on the ability of microorganisms to hydrolyze aesculin.

KIT CONTENT

- 30 Tubes of Aesculin Bile Test

METHOD PRINCIPLE

Many organisms are capable of hydrolyzing the glycoside aesculin to aesculetin and glucose, but only a few of them (e.g. Enterococci, Lancefield Group D streptococci and Lactococci) can do that in the presence of 40% bile. The aesculetin combines with ferric ions in the medium to form a dark brown or black phenolic complex.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: inoculating loop, pipettes, culture media, physiological solution (0.85% saline), quality control organisms.

REAGENTS

- Each tube contains desiccated nutritive components, biochemical substrates (i.e. aesculin and iron salts) and selective agents (i.e. bile and sodium azide).

SPECIMEN

Collect specimens in sterile containers or with sterile swabs and transport to the laboratory. Process each specimen using procedures appropriate for that sample. This product is recommended for use only with pure cultures.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

1. Take the number of Aesculin Bile Test tubes needed from the fridge and allow them to reach room temperature.
2. Add 0.3 mL of physiological solution to each tube.
3. Using a sterile loop, pick up one or more than one well isolated colony from a fresh culture plate (18-24 hours old) to inoculate the test tube.
4. Incubate at $35 \pm 2^\circ\text{C}$ for up to 24 hours.
5. Examine tubes after 4 hours incubation and at regular intervals for broth colour change.

Note: Positive and negative controls should be run simultaneously with the organism to be tested (see QUALITY CONTROL).

INTERPRETING RESULTS

A positive aesculin hydrolysis test is indicated by a colour change to dark brown.

No colour change is considered as a negative reaction.

STORAGE

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

SHELF LIFE

1 year.

QUALITY CONTROL

Control strain		Incubation	Characteristic reactions
<i>Escherichia coli</i>	ATCC® 25922	Up to 24 h/ 35 ± 2°C	Negative reaction: No colour change
<i>Staphylococcus aureus</i>	ATCC® 25923		
<i>Enterococcus faecalis</i>	ATCC® 19433		Positive reaction: Blackening of medium
<i>Enterococcus faecalis</i>	ATCC® 29212		

PERFORMANCE CHARACTERISTICS

Performance testing of Aesculin Bile Test was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

Due to nutritional variation, some strains may result in poor growth or fail to grow on this medium providing false negative results.

H₂S produced by various organisms during metabolism, may react with iron to produce a black complex which interferes with the interpretation leading to false positive results.

WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

ORDER INFORMATION

Product	Packaging	Ref.
Aesculin Bile Test	30 tests	88008

Revision History

Revision	Release Date	Change Summary
0	2023-05-15	Updated layout and content in compliance with IVDR 2017/746, version reset to revision 0

This IFU document and the SDS are available from the online Support Center:

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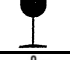
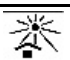
References / Riferimenti

1. Public Health England. (2021). Identification of *Streptococcus* species, *Enterococcus* species and morphologically similar organisms. UK Standards for Microbiology Investigations. ID 4 Issue 4. <https://www.gov.uk/guidance/uk-standards-for-microbiology-investigations-smi-quality-and-consistency-in-clinical-laboratories>
2. EN ISO 11133:2014+Amd1:2018+Amd2:2020. Microbiology of food, animal feed and water – Preparation, production, storage and performance testing of culture media.
3. Public Health England. (2018). Aesculin hydrolysis test. UK Standards for Microbiology Investigations. TP 2 Issue 4.
4. Chuard C, Reller LB (1998) Bile-esculin test for presumptive identification of enterococci and streptococci: effects of bile concentration, inoculation technique, and incubation time. J Clin Microbiol. 36(4):1135-6. DOI: 10.1128/JCM.36.4.1135-1136.1998.
5. Isenberg HD, Goldberg D, Sampson J. (1970) Appl Micro. 20:433-436.

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