Optochine Test

Diagnostic discs for identification of pneumococci.

INTENDED PURPOSE

Test used for differentiating *Streptococcus pneumoniae* from other alpha-haemolytic streptococci. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

DESCRIPTION

Optochine Test are paper discs impregnated with optochin (ethylhydrocupreine hydrochloride) and used in a diagnostic workflow for identification of *Streptococcus pneumoniae* based on susceptibility to that chemical agent.

KIT CONTENT

• 2 cartridges of 50 discs.

Cartridges are individually packed in a foil-sealed blister pack with a desiccant.

METHOD PRINCIPLE

A paper disc impregnated with optochin is applied directly to an inoculated agar plate before incubation. Pneumococci, which are sensitive to optochin, produce a zone of inhibition around the disc, whilst other alphahaemolytic streptococci either grow right up to the edge of the disc or show a very small zone of inhibition.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: inoculating loop, pipettes, culture media, physiological solution, forceps, quality control organisms, ruler/caliper to measure zone sizes.

REAGENTS

• Each disc contains 5 μg of Optochin.

SPECIMEN

Collect specimens in sterile containers or with sterile swabs and transport to the laboratory. Process each specimen using procedures appropriate for that sample. This product is recommended for use only with pure cultures.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

- 1. Allow product to reach room temperature before use for minimizing condensation on the disc.
- 2. Using a sterile loop or swab inoculate evenly the surface of a blood agar plate with a pure culture of the test organism exhibiting the appropriate haemolysis from the primary isolation plate.
- 3. Using sterile forceps, apply one disc of Optochine Test to the surface of the inoculated agar.
- 4. Invert and incubate plates at 35 ± 2 °C for 18-24 hours in atmosphere with 5% CO₂.
- 5. Examine for an inhibition zone around the disc by measuring the diameter with a millimeter ruler or caliper.

INTERPRETING RESULTS

The test organism is considered sensitive (S) to optochin and presumptively *S. pneumoniae* if a zone of inhibition of \geq 14 mm diameter is formed around the disc.

No zone of inhibition (growth up to and around disc) or a zone of inhibition of < 14 mm diameter indicates that the test organism is resistant (R) to optochin and is not *S. pneumoniae*.

STORAGE

Store Optochine Test at -20°C/+8°C in the original packaging. Keep away from sources of heat and avoid excessive changes in temperature. Once opened from the desiccant-containing packaging, discs should be stored at 2-8°C and used within 7 days. Do not use the product beyond the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

SHELF LIFE

2 years.

QUALITY CONTROL

Control strain		Interpretation
Streptococcus pneumoniae	ATCC [®] 6305	S, zone of inhibition of ≥ 14 mm
Streptococcus pyogenes	ATCC [®] 19615	R, no inhibition zone

PERFORMANCE CHARACTERISTICS

Performance testing of Optochine Test was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

Optochine susceptibility is a presumptive test only. Further biochemical and/or serological tests are needed to confirm identification.

Some "viridans" streptococci may produce a small zone of inhibition, i.e. < 14 mm. Occasional strains of optochin resistant *S. pneumoniae* have been reported.

Concentration of CO₂ during incubation may affect the diameters of inhibition zones giving false resistant results. Failure to observe instructions for use may lead to erroneous results.

WARNING AND PRECAUTIONS

- 1) For in vitro diagnostic use (IVD).
- 2) For laboratory professional use only.
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

ORDER INFORMATION

Product	Packaging	Ref.
Optochine Test	2 x 50 discs	9501

Revision History

Revision	Release Date	Change Summary
5	2024-05-13	Added notice to report any malfunction, defect or incident.
4	2023-05-15	Updated layout and content in compliance with IVDR 2017/746.

In case of malfunctions or defects, contact immediately Liofilchem (*) or the local representative.

In case of incident associated with the device, notify immediately Liofilchem (*) or its local representative and the National Competent Authority.

This IFU document and the SDS are available from the online Support Center:

liofilchem.com/ifu-sds

^{*}Please login to https://www.liofilchemstore.it/login.php (user ID and password required) and click on Complaint.

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References / Riferimenti

- 1. Public Health England. (2018). Optochin test. UK Standards for Microbiology Investigations. TP 25 Issue 4. https://www.gov.uk/uk-standards-for-microbiology-investigations-smi-quality-and-consistency-in-clinical-laboratories
- 2. Lennette EH, Balows A, Hausler WJ, Truant JP (1980) Manual of Clinical Microbiology, 3rd ed. ASM, Washington, D.C.
- 3. Bowen MK, Thiele LC, Stearman BD, Schaub IG (1957). J Lab Clin Med. 49:641.
- 4. Bowers EF, Jeffries LR (1955) J Clin Path. 8: 58.
- 5. Moore HF (1915) J Exp Med. 22:269-85.

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