SensiQuattro CANDIDA EU

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SensiQuattro Candida EU

ENGLISH

Panel for determining the susceptibility testing of *Candida* spp. with antimycotic agents.

DESCRIPTION

SensiQuattro Candida EU is a 32-well panel containing 8 desiccated antimycotics at four serial concentrations by doubling.

SensiQuattro Candida EU allows to test the susceptibility of *Candida* spp. to antimycotic agents. The results when correlated with EUCAST (The European Committee on Antimicrobial Susceptibility Testing) breakpoints permit accurate determination of the microorganism as Sensitive (S) or Resistant (R) to antimycotics.

KIT CONTENT

The package contains:

Ref. 76033	20 panels SensiQuattro CANDIDA EU	20 tubes of <i>Physiological Solution</i> (6 ml/tube)
Ref. 79033	4 panels SensiQuattro CANDIDA EU	4 panels of <i>Physiological Solution</i> (6 ml/tube)
1 Instruction sheet		1 Results Data Form

ITEMS NECESSARY BUT NOT INCLUDED IN THE KIT

- Multi-channel pipette 50-300 µl (ref.96759)
- Tips for multi-channel pipette (1000 tips) (ref.96758)
- Solution reservoir for multichannel pipette (ref. 96761)
- Mc Farland 0.5 Barium Sulphate standard (ref.80400)

CONFIGURATION

WELL	ANTIMYCOTIC	CONCENTRATION µg/ml	WELL	ANTIMYCOTIC	CONCENTRATION µg/ml
FLU	Fluconazole	1	AND	Anidulafungin	0.015
FLU	Fluconazole	2	AND	Anidulafungin	0.03
FLU	Fluconazole	4	AND	Anidulafungin	0.06
FLU	Fluconazole	8	AND	Anidulafungin	0.12
VO	Voriconazole	0.06	MIC	Micafungin	0.03
VO	Voriconazole	0.12	MIC	Micafungin	0.06
VO	Voriconazole	0.25	MIC	Micafungin	0.12
VO	Voriconazole	0.5	MIC	Micafungin	0.25
POS	Posaconazole	0.03	FC	Flucytosin	4
POS	Posaconazole	0.06	FC	Flucytosin	8
POS	Posaconazole	0.12	FC	Flucytosin	16
POS	Posaconazole	0.25	FC	Flucytosin	32
CAS	Caspofungin	0.03	AMB	Amphotericin B	0.5
CAS	Caspofungin	0.06	AMB	Amphotericin B	1
CAS	Caspofungina	0.12	AMB	Amphotericin B	2
CAS	Caspofungin	0.25	Growth	Growth control	

Physiological Solution (g\l): Sodium Chloride **9 g**; Distilled Water **1000 ml**; pH **6.8 ± 0.2**

PRINCIPLE OF THE METHOD

The panel is inoculated and rehydrated with a standardised microbial suspension and incubated at 34.5-35.5 °C for 24±2 hours.

After incubation, the susceptibility testing of *Candida* spp. to antimycotic agents is assessed on the basis of growth or inhibition of the microorganisms in media containing antimycotic and a growth indicator.

The **Growth** well does not contain antimycotics; it contains only the culture medium and the indicator.

COLLECTION AND STORAGE OF THE SAMPLES

The *Candida* spp. colonies to be subjected to the susceptibility testing with **SensiQuattro Candida EU** must be taken from a selective or non-selective culture medium used for isolation of the microorganism.

TEST PROCEDURE

1. Take a system **SensiQuattro Candida EU** from its wrapper and let it reach room temperature.
2. Write down name, date and laboratory ID number of the microorganism on the panel.
3. Take several morphologically similar, well-isolated, colonies and suspend them in a tube of Physiological Solution from the kit and obtain a suspension of opacity equivalent to the standard 0.5 McFarland (ref. 80400).
4. Drain the bacterial suspension in the solution reservoir for multi-channel pipetting (ref. 96761).
5. Dispense 0.15 ml of bacterial suspension into each well of the system using an eight tips multichannel pipette or a single tip pipette.
6. Cover the system with the lid provided and incubate at 34.5-35.5 °C for 24±2 hours.

INTERPRETATION OF THE RESULTS

After incubation, note the color changes of the wells and interpret the results using Table no. 2.

The **Growth** well must turn yellow/orange. If its color remains red, the vitality of the inoculated colonies must be checked and the test must be repeated with a new panel and a recently grown microbial culture.

Susceptibility testing results provided by **SensiQuattro Candida EU** must be evaluated according to EUCAST clinical breakpoints¹⁾ and the microorganisms can be interpreted as S (Sensitive) or R (Resistant).

Note results on the Test Result Form (copy as many forms as necessary).

Table no. 2

WELL COLOUR	MICROBIAL GROWTH
Red	Inhibited
Yellow / Orange	Good

QUALITY CONTROL

Every batch of **SensiQuattro Candida EU** is subjected to quality control using the following reference microorganisms:

<i>Candida krusei</i>	ATCC 6258 ³⁾
<i>Candida parapsilosis</i>	ATCC 22019 ³⁾
<i>Candida albicans</i>	ATCC 10231
<i>Candida albicans</i>	ATCC 90028
<i>Candida tropicalis</i>	ATCC 750

FACTORS THAT MAY INVALIDATE THE RESULTS

- Mixed cultures.
- Inaccurate standardisation of the inoculum.
- Use of inoculum solutions other than those suggested.
- Expired (out-dated) panels.
- Temperature and times for incubation other than those suggested.
- Lack of growth in the **Growth** well.

LIMITS AND FURTHER ADVICE

The results obtained with the system **SensiQuattro Candida EU** must be interpreted by a clinical microbiologist and physician before proceeding to therapeutic treatment of the patient.

PRECAUTIONS

The product, **SensiQuattro Candida EU**, is not classified as hazardous under current legislation, however see the safety data sheet for a correct use.

SensiQuattro Candida EU is a disposable device to be used only for diagnostic use *in vitro*.

It must be used in the laboratory by appropriately trained personnel, using approved aseptic and safety methods for handling pathogenic agents.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the storage or transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, do not alter in any way the performance of the product. Discard the panel if there are visible signs of deterioration.

DISPOSAL OF USED MATERIAL

After use, **SensiQuattro Candida EU** and material that has come into contact with the sample must be decontaminated and disposed of in accordance with guidelines used in the laboratory for decontamination and disposal of potentially infected material.

REFERENCES

1. Antimicrobials for Candida infections - EUCAST clinical MIC breakpoints 2011-04-27 (v. 3.0).
2. EUCAST Definitive Document Edef 7.1: method for the determination of broth dilution MICs of antifungal agents for fermentative yeasts. Subcommittee on Antifungal Susceptibility testing (AFST) of the ESCMID European Committee for Antimicrobial Susceptibility Testing (EUCAST). *Clin Microbiol infect* 2008; 14:398-405.
3. Multicentre determination of quality control strains and quality control ranges for antifungal susceptibility testing of yeasts and filamentous fungi using the methods of the Antifungal Susceptibility Testing Subcommittee of the European Committee on Antimicrobial Susceptibility Testing (AFST-EUCAST). M.Cuenca-Estrella et al. *Clin Microbiol infect* 2007; 13:1018-1022.

PRESENTATION

Product	Ref.	Packaging
SensiQuattro Candida EU	76033	20 tests
SensiQuattro Candida EU	79033	4 tests

TABLE OF SYMBOLS

IVD <i>In Vitro</i> Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
REF Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	LOT Batch code