MIF COLOR KIT

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INSTRUCTION FOR USE

1. Take 1 ml of B solution and transfer it into the A solution container.
2. Take with apposite loop a little quantity of faeces and crush it against the internal side of flask
3. Close up the lid and mix until the homogeneous suspension of sample.
4. Leave to replace for 1 hour.
5. Take with a pipette a little quantity of sample and to put down a drop on microscope slide
6. Cover the microscope slide and examine at microscope.

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INSTRUCTION FOR FAECAL SAMPLES COLLECTION

1. The patient must be suspend any therapy with antiacid, antidiarrhoic, antimicrobial and laxative drug.
2. The faecal samples must be collected in clean, large container or on clean paper sheet; do not utilize faeces from toilettes or mixing with urine.
3. Send the sample to laboratory within 1 hour to emission

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CONTENT OF KIT

50 Bottles of A Solution
1 Bottle of B Solution

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BIBLIOGRAPHY

1. J.W.Smith and M.S. Bartlett. "Diagnostic parasitology. Introduction and methods", **55**, p.595-611. In E.H.Lennette, A.Balows, W.J.Hausler J.r., H.J.Shadomy (ed) Manual of clinical microbiology, Fourth edition American society for Microbiology, Washington, D.C. 1985.
2. D.M. Melvin and G.R.Healy "Intestinal and urogenital protozoa", **57**, p.631-350. In E.H.Lennette, A.Balows, W.J.Hausler J.r., H.J.Shadomy (ed) Manual of clinical microbiology, Fourth edition American society for Microbiology, Washington, D.C. 1985.
3. L.R.Ash and T.C.Orihel "Intestinal helminths", **59**, p.660-673. In E.H.Lennette, A.Balows, W.J.Hausler J.r., H.J.Shadomy (ed) Manual of clinical microbiology, Fourth edition American society for Microbiology, Washington, D.C. 1985.

Liofilchem s.r.l Bacteriology Products

Via Scozia-Zona industriale - 64026 Roseto degli Abruzzi Tel. +39.085.8930745 - Fax +39.085.8930330
Web site: <http://www.liofilchem.net> E-mail: liofilchem@liofilchem.net