BRUCELLA Supplement

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BIBLIOGRAFIA

- Moyer, N.P., and L.A. Holcomb (1995). *Brucella*, p. 549-555.
- Vanderzant, C., and D.F. Splittstoesser (ed.) (1992). *Compendium of methods for the microbiological examination of food*, 3rd ed. American Public Health Association, Washington, D.C.
- Alton, Jones (1968). *La brucellosi techniques de laboratoire*. Geneve: OMS.

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| | | |
|----------------------------|--------------|-------------------|
| Prodotto | REF | Σ |
| BRUCELLA Supplement | 81003 | 10 flaconi |

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Rev.0 / 06.04.2005

BRUCELLA Supplement

Selective supplement for the isolation of *Brucella* spp.

DESCRIPTION

BRUCELLA Supplement is a selective supplement for the isolation of *Brucella* spp from milk. The selective supplement is made of a freeze-dried antibiotics mixture: Polymyxin B, Bacitracin, Cicloheximide, Nalidixic acid, Nystatin, Vancomycin. BRUCELLA Supplement is used for selective enrichment of culture media Brucella Agar Base code 610079 or 620079, Columbia Agar Base code 610013 or 620013, Blood Agar Base N.2 code 610188 or 620188.

KIT CONTENTS

Each kit contains:

- 10 bottles of freeze-dried BRUCELLA Supplement
- 1 Instruction sheet

PRINCIPLE OF THE METHOD

The slow growth of *Brucella* species requires the presence of selective agents inside medium that avoid excessive growth of contaminant microorganisms present in the sample in examination. Polymyxin B is a broad-spectrum antibiotic, Bacitracin interferes with the synthesis of bacterial wall and it is active mostly against Gram-positive bacteria, Cicloheximide and Nystatin have antimicotic activity, Nalidixic acid is an active quinolone against Gram-negative bacteria, Vancomycin inhibits the synthesis of cell wall in Gram-positive bacteria.

COMPOSITION

| BRUCELLA Supplement | | |
|---------------------|-------------------|------------------------|
| | Contents / bottle | Contents / l of medium |
| Polymyxin B | 2500 UI | 5000 UI |
| Bacitracin | 12500 UI | 25000 UI |
| Cicloheximide | 50,0 mg | 100,0 mg |
| Nalidixic acid | 2,5 mg | 5,0 mg |
| Nystatin | 50000 UI | 100000 UI |
| Vancomycin | 10,0 mg | 20,0 mg |

PROCEDURE FOR USE

1. Aseptically reconstitute the contents of one bottle of BRUCELLA Supplement with 10 ml of a solution of methanol and sterile distilled water in the ratio 1:1. Incubate at 36 ± 1 °C for 10-15 minutes. Shake until completely dissolved, avoiding foam formation.
2. Aseptically add the entire contents of one bottle (10 ml) to 500 ml of Brucella Agar Base medium code 610079 or 620079, Columbia Agar Base medium code 610013 or 620013, Blood Agar Base N.2 medium code 610188 or 620188, autoclaved, cooled at 45-50 °C with the addition of 10% of inactivated horse serum and 1-5% of sterile glucose solution.
3. Mix with care.
4. Distribute into Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical documentation for Brucella Agar Base medium code 610079 or 620079, or Columbia Agar Base medium code 610013 or 620013, or Blood Agar Base N.2 medium code 610188 or 620188.

QUALITY CONTROL

1. Control of the appearance: freeze-dried product, colour orange.
2. Microbiological control.

Prepare the plates using as base BRUCELLA AGAR BASE code 610079 or 620079 supplemented with 5% of inactivated horse serum and 1% of sterile glucose solution, and enriched with BRUCELLA Supplement (1 bottle in 500 ml of medium). Plates are inoculated with the strains indicated in the microbiological control table.

Incubation conditions: 48 h at 36 ± 1 °C, in atmosphere with 5-10% of CO₂.

Microbiological control

| Control strains | | Growth |
|------------------------------|------------|-----------|
| <i>Brucella abortus</i> | ATCC 4315 | Good |
| <i>Staphylococcus aureus</i> | ATCC 25923 | Inhibited |
| <i>Escherichia coli</i> | ATCC 25922 | Inhibited |

PRECAUTIONS

The product BRUCELLA Supplement is classifiable as hazardous under current legislation; it is recommended that the Safety Data Sheet be consulted on its use.

BRUCELLA Supplement is a selective supplement to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store BRUCELLA Supplement at 2-8 °C in its original packaging. In such conditions BRUCELLA Supplement will remain valid until the expiry date indicated on the label. Do not use beyond that date. Eliminate without using if there are signs of deterioration.

REFERENCES

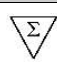
- Moyer, N.P., and L.A. Holcomb (1995). *Brucella*, p. 549-555.
- Vanderzant, C., and D.F. Splittstoesser (ed.) (1992). *Compendium of methods for the microbiological examination of food*, 3rd ed. American Public Health Association, Washington, D.C.
- Alton, Jones (1968). *La brucellose techniques de laboratoire*. Geneve: OMS.

PRESENTATION

| Product | REF | Σ |
|---------------------|-------|------------|
| BRUCELLA Supplement | 81003 | 10 bottles |

One bottle is sufficient to prepare 500 ml of medium

TABLE OF SYMBOLS

| | | | | |
|---|---|--|--|--|
| IVD In Vitro Diagnostic Medical Device |  Do not reuse |  Manufacturer |  Contains sufficient for <n> tests |  Temperature limitation |
| REF Catalogue number |  Fragile, handle with care |  Use by |  Caution, consult accompanying documents | LOT Batch code |

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