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BIBLIOGRAFIA

- MacFaddin, J.D. (1985). Media for isolation-cultivation-identification-identification-maintenance of medical bacteria, p. 695-699, vol. 1. Williams & Wilkins, Baltimore, MD.
- Schaedler, R.W., R. Dubos, and R. Costello. (1965). The development of bacterial flora in the gastrointestinal tract of mice. J. Exp. Med. **122**: 59.

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Prodotto	REF	Σ
SCHAEDLER <i>Supplement</i>	81054	10 flaconi

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Rev.1 / 21.01.2008

SCHAEDLER Supplement

Selective supplement for the preparation of SCHAEDLER AGAR BASE for Gram-negative anaerobe bacteria isolation.

DESCRIPTION

SCHAEDLER *Supplement* is a freeze-dried selective supplement constituted by Kanamycin and Vancomycin, to use as a supplement of the culture medium SCHAEDLER AGAR BASE cod. 610043 or 620043 for Gram-negative anaerobe bacteria isolation.

KIT CONTENTS

Each kit contains:

- 10 vials of SCHAEDLER *Supplement* freeze-dried.
- 1 instructions sheet

PRINCIPLE OF THE METHOD

SCHAEDLER *Supplement* supports Gram-negative anaerobe bacteria isolation thanks to the selective activity of the antibiotics: Kanamycin has a wide spectrum of action which includes Gram-positive and Gram-negative bacteria, binds to 70S ribosomal subunit and inhibits translocation; Vancomycin is a glycopeptide which inhibits Gram-positive bacteria growth interfering with cell wall synthesis.

COMPOSITION

SCHAEDLER Supplement		
	Content / vial	Content / l of medium
Kanamycin	50.0 mg	100.0 mg
Vancomycin	3.75 mg	7.5 mg

PROCEDURE FOR USE

1. Aseptically reconstitute the content of one vial of SCHAEDLER *Supplement* with 5 mL of sterile distilled water.
2. Mix to complete dissolution and add to 500 mL of SCHAEDLER AGAR BASE cod. 610043 or 620043, autoclaved, cooled to 45-50°C and added with 5% of defibrinated sheep blood and with 0.5 mL of Vitamin K 1% *Supplement* (cod. 80053).
3. Mix well and dispense in Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical documentation for SCHAEDLER AGAR BASE cod. 610043 or 620043.

QUALITY CONTROL

1. Control of the appearance: freeze-dried yellow in colour.
2. Microbiological control.

Prepare the plates using as base SCHAEDLER AGAR BASE cod. 610043 or 620043 added with the content of 1 vial of SCHAEDLER *Supplement* (1 vial in 500 mL of medium). The plates are inoculated with the strains indicated in the microbiological control table. Incubation conditions: 36 ± 1 °C for 18-48 hours. Microbiological control:

	Control strains	Growth
<i>Clostridium perfringens</i> *	ATCC 13124	Good
<i>Streptococcus pyogenes</i>	ATCC 19615	Inhibited
<i>Escherichia coli</i>	ATCC 25922	Inhibited
<i>Staphylococcus aureus</i>	ATCC 25923	Inhibited

* in CO₂

PRECAUTIONS

SCHAEDLER *Supplement* contains dangerous substances according to directives 1999/45/CE and 2001/60/CE. SCHAEDLER *Supplement* is a selective supplement to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store SCHAEDLER *Supplement* at 2-8 °C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration. Once reconstituted, the product can be stored for a maximum duration of 30 days at -20°C, shielded from light.

REFERENCES

- MacFaddin, J.D. (1985). Media for isolation-cultivation-identification-maintenance of medical bacteria, p. 695-699, vol. 1. Williams & Wilkins, Baltimore, MD.
- Schaedler, R.W., R. Dubos, and R. Costello. (1965). The development of bacterial flora in the gastrointestinal tract of mice. J. Exp. Med. **122**: 59.

PRESENTATION

Product	REF	
SCHAEDLER <i>Supplement</i>	81054	10 vials

One vial is sufficient to prepare 500 mL of medium

TABLE OF SYMBOLS

 In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	 Batch code

LIOFILCHEM Bacteriology Products

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745 Fax +390858930330 Website: www.liofilchem.net E-Mail: liofilchem@liofilchem.net

Rev.1 / 21.01.2008