Chromatic™ Staph aureus Supplement

Selective supplement for the isolation of *Staphylococcus aureus*.

DESCRIPTION

Chromatic™ Staph aureus Supplement is a selective lyophilized supplement used for the preparation of Chromatic™ Staph aureus (ref. 610616, 620616). The complete medium is used for the isolation and differentiation of *S. aureus* from clinical and nonclinical specimens.

KIT CONTENTS

Each kit contains:

- 10 vials of freeze-dried Chromatic™ Staph aureus Supplement
- an instruction sheet

PRINCIPLE OF THE METHOD

Chromatic™ Staph aureus Supplement is a chromogenic and antibiotic mix that inhibits the growth of most organisms other than staphylococci enhancing bacterial differentiation as well.

COMPOSITION

	Content / vial	Content / liter of medium
Chromogenic and Selective Mix	0.4 g	0.8 g

PROCEDURE FOR USE

1. Aseptically reconstitute the content of one vial of Chromatic™ Staph aureus Supplement with 5 ml sterile distilled water.
2. Mix to complete dissolution and add to 500 ml Chromatic™ Staph aureus, autoclaved and cooled to 45-50°C.
3. Mix well and dispense in Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical documentation of the medium being prepared.

QUALITY CONTROL

1. Visual inspection: pinkish button, limpid pinkish solution once reconstituted.
2. Microbiological control.
Prepare the medium per label directions. Inoculate the plates with the strains indicated below and incubate at 35±2°C for 18-24 hours.

Control strains		Growth	Colony color
<i>Staphylococcus aureus</i>	ATCC® 6538	Good	Pink-purplish
<i>Staphylococcus xylosus</i>	ATCC® 29971	Good	Greenish
<i>Escherichia coli</i>	ATCC® 25922	Inhibited	---

PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous; it is nevertheless recommended that the Safety Data Sheet be consulted on its use. The product is a selective supplement to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration. Once reconstituted, the product can be stored for a maximum duration of 30 days at -20°C, shielded from light.

REFERENCES

- O. Gaillot. Evaluation of CHROMagar Staph aureus, a new chromogenic medium, for isolation and presumptive identification of *Staphylococcus aureus* from human clinical specimens. Laboratoire de Bacteriologie-Virologie, Hopital Necker-Paris, France, rapport de Septembre 1999.
- Carricao A., A.M. Freydiere et al. Performance of the Chromogenic medium CHROMagar Staph aureus and the Staphylochrom Coagulase Test in the detection and identification of *Staphylococcus aureus* in clinical specimens. Journal of clinical microbiology, July 2001, P.2581-2583, Vol:39 N°7.

PRESENTATION

Product	Ref.	Contents
Chromatic™ Staph aureus Supplement	81085	10 vials

One vial is sufficient to prepare 500 ml of medium.

TABLE OF SYMBOLS

IVD In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
REF Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	LOT Batch code

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