Chromatic™ ESB�+AmpC Supplement

Selective supplement for detection of Extended-spectrum β -lactamase and AmpC producers.

DESCRIPTION

Chromatic™ ESB�+AmpC Supplement is a selective lyophilized supplement used for the preparation of Chromatic™ ESB� Agar Base (Ref. 610629, 620629). The complete medium is used for detection of ESB� and AmpC in Enterobacteriaceae directly from clinical specimens.

KIT CONTENTS

Each kit contains:

- 10 vials of lyophilized Chromatic™ ESB�+AmpC Supplement
- 1 instructions sheet

PRINCIPLE OF THE METHOD

Selective mix allows the optimal recovery of ESB� and AmpC producers while inhibiting other organisms.

COMPOSITION

	Content / vial	Content / liter of medium
Selective Mix	0.5 g	1.0 g

PROCEDURE FOR USE

1. Reconstitute aseptically the content of one vial of Chromatic™ ESB�+AmpC Supplement with 5 ml of sterile distilled water
2. Mix to complete dissolution and add aseptically to 500 ml of Chromatic™ ESB� Agar Base autoclaved and cooled at 45-50°C.
3. Mix with care and pour into Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical sheet of the medium being prepared.

QUALITY CONTROL

1. Visual inspection: whitish button, limpid solution once reconstituted.
2. Microbiological control.
Prepare the medium per label directions. Inoculate the plates with the microbial strains indicated below and incubate at at 35±2°C for 18-24 h.

Control strains		Growth	Colony color
<i>Klebsiella pneumoniae</i> (ESBL positive, AmpC negative)	ATCC® 700603	Good	Green-blue
<i>Klebsiella pneumoniae</i> (ESBL negative, AmpC positive)	ATCC® BAA-1144	Good	Green-blue
<i>Escherichia coli</i> (ESBL negative, AmpC negative)	ATCC® 25922	Inhibited	---

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is intended for *in vitro* Diagnostic Use only and must be used by properly trained operators.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the storage or transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, do not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

REFERENCES

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
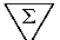
- EUCAST guidelines for detection of resistance mechanisms and specific resistances of clinical and/or epidemiological importance. Version 1.0, 2013.
- Podschun R, Ullman U. *Klebsiella* spp as Nosocomial Pathogens: Epidemiology, Taxonomy, Typing Methods, and Pathogenicity Factors. *Clinical Microbiology Reviews*. 1998; 11 (4): 589-603.
- Geiss H.K. Comparison of two test kits for rapid identification of *Escherichia coli* by a beta-glucuronidase assay. *European Journal of Clinical Microbiology & Infections Diseases*. 1990; 9 (2):151-152.

PRESENTATION

Product	Ref.	Content
Chromatic™ ESB�+AmpC Supplement	81090	10 vials

One vial is sufficient to prepare 500 ml of medium.

TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	 Do not reuse

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BIBLIOGRAFIA

- EUCAST guidelines for detection of resistance mechanisms and specific resistances of clinical and/or epidemiological importance. Version 1.0, 2013.
- Podschun R, Ullman U. *Klebsiella* spp as Nosocomial Pathogens: Epidemiology, Taxonomy, Typing Methods, and Pathogenicity Factors. *Clinical Microbiology Reviews*. 1998; 11 (4): 589-603.
- Geiss H.K. Comparison of two test kits for rapid identification of *Escherichia coli* by a beta-glucuronidase assay. *European Journal of Clinical Microbiology & Infectious Diseases*. 1990; 9 (2):151-152.

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LIOFILCHEM® S.r.l.