Chromatic GBS Supplement

Selective and chromogenic supplement for detection of *Streptococcus agalactiae*.

DESCRIPTION

Chromatic GBS Supplement is a lyophilized supplement used in combination with Tween 20 (ref. 80032) for the preparation of Chromatic GBS (ref. 610639). The complete medium is used for the isolation and differentiation of *S. agalactiae*.

KIT CONTENTS

Each kit contains:

- 10 vials of freeze-dried Chromatic GBS Supplement

An instruction sheet is available from the online Support Center: liofilchem.com/ifu-sds

PRINCIPLE OF THE METHOD

Chromatic GBS Supplement contains chromogenic substrates for colony identification and antibiotics for inhibition of competing flora.

PROCEDURE FOR USE

1. Aseptically reconstitute the contents of one vial of Chromatic GBS Supplement with 5 ml sterile distilled or deionized water.
2. Mix to complete dissolution and add to 500 ml of Chromatic GBS, autoclaved as directed and cooled to 45-50°C;
3. Mix well and pour into sterile Petri dishes.

NOTES:

- It is advisable to add the other supplement (1.5 ml Tween 20 to 500 ml base medium) before autoclaving at 104°C for 10 min.
- The selective supplement is heat-sensitive and must be added after the sterilization.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical sheet of the medium being prepared.

QUALITY CONTROL

1. Control of the appearance: freeze-dried white in colour.
2. Microbiological control.
Prepare the medium per label directions. Inoculate the medium with the strains indicated below and incubate at $35 \pm 2^\circ\text{C}$ for 21 ± 3 h under aerobic atmosphere.

Control strains

Specification

<i>Streptococcus agalactiae</i>	ATCC® 13813	Good growth, mauve to red colonies
<i>Enterococcus faecalis</i>	ATCC® 19433	Good growth, blue to purple colonies
<i>Escherichia coli</i>	ATCC® 25922	Inhibition

WARNING AND PRECAUTIONS

For professional use only. Operators must be trained and have certain experience in the laboratory methods. Please read the instructions carefully before using this product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.

Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.

STORAGE

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

REFERENCES

1. Public Health England (2021) UK SMI ID 4 issue 4: Identification of *Streptococcus* species, *Enterococcus* species and morphologically similar organisms. https://assets.publishing.service.gov.uk/government/uploads/system/uploads/attachment_data/file/1019677/ID_4i4.pdf

PRESENTATION

Product	Ref.	Contents
Chromatic GBS Supplement	81103	10 vials

One vial is sufficient to prepare 500 ml of medium.

TABLE OF SYMBOLS

 Batch code	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Consult instructions for use	

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LIOFILCHEM® S.r.l.