Methyl Red Droppers

ENGLISH

Reagent for the Methyl Red test,
in dropper bottles.

DESCRIPTION

Methyl Red is a pH indicator used to perform the Methyl Red (MR) test for the differentiation of members of the Enterobacteriaceae family.

KIT CONTENTS

- 20 bottles of Methyl Red Droppers.
- Instructions sheet.

METHOD PRINCIPLE

Some bacteria utilize glucose with production of a stable acid like lactic acid, acetic acid or formic acid as end product resulting in a low pH and colour change of methyl red from yellow to red (MR positive). Although all Enterobacteriaceae ferment glucose, MR negative organisms use a different metabolic pathway which results in an alkaline or less acidic reaction (no red colour) due to the production of acetyl-methyl carbinol (acetoin), a neutral reacting end product.

COMPOSITION

Methyl red 0.02 g, ethanol 95% 60 ml, distilled water 40 ml (per 100 ml).

TEST PROCEDURE

1. Prior to proceed with the following method, allow MR-VP medium (ref. 24149) and dropper bottle to equilibrate to room temperature.
2. Lightly inoculate the medium with a pure culture of the organism in question.
3. Incubate aerobically at 35°C for 2-5 days.
4. Following an incubation period of 48 hours at least, aliquot 2.5 ml of the broth to a clean test tube.
5. Add five drops of methyl red reagent.

INTERPRETATION OF RESULTS

A positive MR test is given by the immediate development of a red colour on the surface of the medium.

A negative MR test is demonstrated by the development of a yellow colour.

QUALITY CONTROL FOR THE USER

MR-VP medium, 48 h incubation at 35°C under aerobic atmosphere.

Control strains		MR test
<i>Escherichia coli</i>	ATCC® 25922	Positive
<i>Enterobacter cloacae</i>	ATCC® 13047	Negative

PRECAUTIONS

Methyl Red Droppers is classifiable as hazardous under current legislation; it is recommended that the Safety Data Sheet be consulted on its use. The product is intended for *in vitro* diagnostic use and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the storage or transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, do not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

ELIMINATING USED MATERIAL

After use, used Methyl Red Droppers and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

BIBLIOGRAPHY

1. MacFaddin, J.F. (1985) Media for Isolation, Cultivation, Identification, Maintenance of Bacteria, Vol. I. Williams & Wilkins, Baltimore, MD.
2. Lennette, E.H., A. Balows, W.J. Hausler, Jr., and H.J. Shadomy (ed.). (1985) Manual of clinical microbiology, 4th ed. American Society for Microbiology, Washington, D.C.
3. Finegold, S.M., and W.J. Martin. (1982) Bailey and Scott's diagnostic microbiology, 6th ed. The C.V. Mosby Co., St. Louis.

PRESENTATION

Product	Ref.	Content
Methyl Red Droppers	87009	20 x 2.5 ml bottles

TABLE OF SYMBOLS

 LOT	Batch code	 IVD	In vitro Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
 REF	Catalogue number		Temperature limitation		Contains Sufficient for <n> tests		Caution, consult accompanying documents		Do not reuse

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LIOFILCHEM® S.r.l.