Shigella antiserum

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E-Mail: liofilchem@liofilchem.net

Shigella antiserum

**Lyophilized polyvalent serum for direct serological characterization of *Shigella* spp.
and for confirmation of the isolation on COPRO SYSTEM plus**

ENGLISH

DESCRIPTION

Shigella antiserum is an agglutinating, polyvalent, serum for direct serological characterization of species of *Shigella* and for confirmation of the isolation on **COPRO SYSTEM plus**.

Shigella antiserum is used non-diluted exclusively in agglutination test on slide.

KIT CONTENTS

Each kit contains:

- the polyvalent serum
- 1 dropper
- 1 instructions sheet

ITEMS NECESSARY BUT NOT INCLUDED IN THE KIT

Sundry material used in microbiology laboratories and suitable culture media.

COMPOSITION

Each bottle contains lyophilized polyvalent serum. The test serum is absorbed serum from immunised rabbits.

PREPARATION

1. Reconstitute the content of the bottle with 2 mL of sterile distilled water.
2. Gently mix the content of the bottle.
3. Reconstituted serum, stored at 2-8°C, can be used at least for 12 months.

TEST PROCEDURE

Agglutination test for serological characterization of *Shigella* spp. can be performed directly with isolated colonies on agar medium, or drawing the suspension from the well **11-SHI** of **COPRO SYSTEM plus** (cod. 71675).

On an agglutination slide dissolve a loopful of the pure bacterial growth to characterize in 1 drop (35 µL) of physiological solution, or put 1 drop of bacterial suspension taken from the well **11-SHI** of **COPRO SYSTEM plus**. Add 1 drop of the agglutinating serum and, after mixing thoroughly with a glass stick, rotate the slide observing for the agglutination for 2-3 minutes. It is advisable to prepare a slide without serum, substituting it with physiological solution, to control the homogeneity of the bacterial suspension.

In case of positive reaction a bacterial agglutination is particularly evident observing the slide on a source of light.

Belated agglutination (more than 3 minutes) is not significant for diagnosis because it could not be specific.

TEST PERFORMANCE

Shigella antiserum is a polyvalent serum prepared with the following types of Shigella:

- Sh. sonnei (forms S and F)
- Sh. flexneri (type-specific antigens 1 to 6 and group-specific antigens 3,4(y); 6; 7,8(x))
- Sh. dysenteriae (types 1 to 10)
- Sh. boydii (types 1 to 15)

PRECAUTIONS

The product **Shigella antiserum** is not classified as hazardous under current legislation; in any case it is recommended that the safety data sheet be consulted for its use.

Shigella antiserum is a supplement to be used only for *in vitro* diagnostic use. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE

Store **Shigella antiserum** at 2-8°C in its original package. In such conditions the product is valid until the expiry date shown on the label. Do not use them beyond that date. Dispose of them if they show signs of deterioration.

DISPOSAL OF USED MATERIAL

After the use, **Shigella antiserum** and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

BIBLIOGRAFIA

1. Holt, J.G., N.R. Krieg, P.H. Sneath, J.T. Staley and S.T. Williams. 1994. *Bergey's manual of determinative bacteriology*, 9th ed. Williams & Wilkins, Baltimore, Md.

PRESENTATION

Description	Code	Packaging
Shigella antiserum	96148	1 bottle

TABLE OF SYMBOLS

IVD In Vitro Diagnostic Medical Device	Do not reuse	Manufacturer	Contains sufficient for <n> tests	Temperature limitation
REF Catalogue number	Fragile, handle with care	Use by	Caution, consult accompanying documents	LOT Batch code

LIOFILCHEM s.r.l.

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745 Fax +390858930330 Website: www.liofilchem.net

Rev.2 / 15.09.2008

E-Mail: lioilchem@lioilchem.net