## BIGGY NICKERSON AGAR

Selective and differential medium used in the detection, isolation and presumptive identification of *Candida* species.

### TYPICAL FORMULA (g/l)

Yeast Extract	1.0
Ammonium Chloride	2.5
Bismuth Citrate	2.5
Sodium Sulphite	3.0
Glucose	10.0
Glycine	10.0
Agar	18.0
Final pH 6.8 ± 0.2	

### DESCRIPTION

**BIGGY NICKERSON AGAR** is a selective and differential medium used in the detection, isolation and presumptive identification of *Candida* species.

### PRINCIPLE

Yeast extract is a source of aminoacids and vitamins of group B. Glucose is a source of energy. Potato starch and dextrose support luxuriant growth of fungi. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the plate by streaking the specimen on the surface of the medium, using a sterile loop. Incubate at 25-30°C for 18-48 hours.

### INTERPRETATION of RESULTS

Observe for the growth of colonies.

### STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

### WARNING and PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of ≥1%. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL of WASTE

Disposal of waste must be carried out according to national and local regulations in force.

### REFERENCES

1. Nickerson. 1953. J. Infect. Dis. 93:43.
2. Haley, Trandel and Coyle. 1980. Cumitech 11, Practical methods for culture and identification of fungi in the clinical mycology laboratory. Coord. ed., Sherris. American Society for Microbiology, Washington, D.C.
3. Isenberg (ed.). 1992. Clinical microbiology procedures handbook, vol. 1. American Society for Microbiology, Washington, D.C.

**Liofilchem s.r.l.**

Via Scozia, Zona industriale - 64026 Roseto degli Abruzzi (Te) Italy - Tel. +39.0858930745 - Fax +39.0858930330  
Web site: <http://www.liofilchem.net> - E-mail: [liofilchem@liofilchem.net](mailto:liofilchem@liofilchem.net)

## PRODUCT SPECIFICATIONS

### NAME

**BIGGY NICKERSON AGAR**

### PRESENTATION

Ready plates (90 mm) containing 22+/-1 ml of medium.

### STORAGE

10-25°C

### PACKAGING

Code	Content	Packaging
10021	20 plates	<ul style="list-style-type: none"> <li>5 plates in thermically soldered film</li> <li>4 x 5 plates in cardboard box</li> </ul>
10021*	100 plates	<ul style="list-style-type: none"> <li>5 plates in thermically soldered film</li> <li>2 x 5 plates in thermically soldered bag</li> <li>10 piles (2 x 5 ps) in box</li> </ul>

### pH OF THE MEDIUM

6.8 ± 0.2

### USE

**BIGGY NICKERSON AGAR** is a selective and differential medium used in the detection, isolation and presumptive identification of *Candida* species.

### TECHNIQUE

Refer to technical sheet of the product.

### APPEARANCE of the MEDIUM

Cream yellow medium, opalescent.

### SHELF LIFE

6 months

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
  - 7 days at 25 ± 1°C, in aerobiosis
  - 7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
  - Inoculum for productivity: 10-100 UFC/ml
  - Inoculum for specificity: ≤ 10<sup>4</sup> UFC/ml
  - Incubation conditions: 18-48 h at 25-30°C, in aerobiosis

Microorganism		Growth
<i>Candida albicans</i>	ATCC 10231	Good
<i>Saccharomyces cerevisiae</i>	ATCC 9763	Good
<i>Escherichia coli</i>	ATCC 25922	Inhibited

### TABLE OF SYMBOLS

<b>IVD</b> In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
<b>REF</b> Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	<b>LOT</b> Batch code

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