ROGOSA AGAR

Selective medium for the isolation and the enumeration of lactobacilli.

TYPICAL FORMULA (g/l)

Tryptone.....	10.0
Yeast Extract.....	5.0
Glucose.....	10.0
Arabinose.....	5.0
Sucrose.....	5.0
Sodium Acetate.....	15.0
Monopotassium Phosphate.....	6.0
Ammonium Citrate.....	2.0
Magnesium Sulphate.....	0.57
Manganese Sulphate.....	0.12
Iron Sulphate.....	0.03
Agar.....	15.0
Tween 80.....	1.0
Final pH	5.4 ± 0.2

DESCRIPTION

ROGOSA AGAR is a selective medium for the isolation and the enumeration of lactobacilli from oral, vaginal and faecal samples.

PRINCIPLE

Tryptone provides carbon and nitrogen. Yeast extract is a source of vitamins and amino acids. Glucose, arabinose and sucrose are a source of carbohydrates and so of carbon. Sodium acetate and ammonium citrate, as the acid pH, inhibit streptococci, moulds and other microorganisms which are present in the oral flora. Monopotassium phosphate forms the buffering system of the medium. Magnesium sulfate, manganese sulfate and iron sulphate are a source of inorganic ions. Tween 80 acts as surfactant. Agar is the solidifying agent.

TECHNIQUE

Inoculate streaking the sample to examine onto the surface of the medium in order to isolate pure colonies from samples containing a mixed flora. Incubate at 36+/-1 °C for 40-48 hours in microaerophilic conditions. Thermophilic lactobacilli must be incubated at 42°C for 48 hours, mesophilic-psycrophilic organisms at 30°C for 2 days and at 22°C for another day, meat leuconostoc is incubated at 25°C for 3 days.

INTERPRETATION of RESULTS

Observe for the growth of colonies.

STORAGE

10-25 °C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING and PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of 1%. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL of WASTE

Disposal of waste must be carried out according to national and local regulations in force.

REFERENCES

1. Rogosa, M., J.A. Mitchell, and R.F. Wiseman, (1971) *J. Bacteriol.* 62: 132.
2. Rogosa, M., J.A. Mitchell, and R.F. Wiseman, (1951). *J. Dental Res.* 30 : 682.
3. ISO/TC34/SC6/WG15. (1984). *Enumeration of Lactobacteriaceae in meat and meat products.*
4. Sharpe M.E. (1960) *Lab. Practice* 9: 223-227.

PRODUCT SPECIFICATIONS

NAME

ROGOSA AGAR

PRESENTATION

90 mm ready-to-use plates containing 22±1 ml of medium

STORAGE

10-25°C

PACKAGING

Code	Content	Packaging
10039	20 plates	<ul style="list-style-type: none"> • 5 plates in thermally soldered film • 4 x 5 plates in cardboard boxes
10039*	100 plates	<ul style="list-style-type: none"> • 5 plates in thermally soldered film • 2 x 5 plates in thermally soldered bag • 10 piles (2 x 5 ps) in box

pH of the MEDIUM

5.4 ± 0.2

USE

ROGOSA AGAR is a selective medium for the isolation and the enumeration of lactobacilli from oral, vaginal and fecal samples.

TECHNIQUE

Refer to technical sheet of the product.

APPEARANCE of the MEDIUM

Amber medium, slightly opalescent.

SHELF LIFE

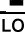
6 months

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
 - 7 days at 25 ± 1°C, in aerobiosis
 - 7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
 - Inoculum for productivity: 10-100 UFC/ml
 - Inoculum for selectivity : 10⁴-10⁵ UFC/ml
 - Inoculum for specificity: ≤ 10⁴ UFC/ml
 - Incubation conditions: 40-48 hours at 36 ± 1°C in microaerophilia

Microorganism		Growth
<i>Lactobacillus delbrueckii</i>	ATCC 4797	Good
<i>Staphylococcus aureus</i>	ATCC 25923	Inhibited

TABLE OF SYMBOLS

Symbol	Meaning
REF	Catalogue number
IVD	<i>In vitro</i> Diagnostic Medical Device
	Manufacturer
	Temperature limitation
	Kit content
	Use by
LOT	Batch code
	Consult accompanying documents

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