## WILKINS CHALGREN AGAR

Medium for culture and susceptibility test of anaerobes.

### TYPICAL FORMULA (g/l)

Peptone.....	10.0
Tryptone.....	10.0
Sodium Chloride.....	5.0
Yeast Extract.....	5.0
Glucose.....	1.0
L- Arginine.....	1.0
Sodium Pyruvate.....	1.0
Agar.....	15.0
Hemin.....	0.005
Vitamin K1.....	0.0005
Final pH 7.1 ± 0.1	

### DESCRIPTION

WILKINS CHALGREN AGAR is used for susceptibility testing of anaerobes and for isolating and cultivating anaerobes.

### PRINCIPLE

Peptone and tryptone provide nitrogen and amino acids. Yeast extract is the vitamin source. Sodium chloride maintains the osmotic balance of the medium. Glucose is the carbon source. Agar is the solidifying agent. Horse serum provides essential growth factors. L-arginine and sodium pyruvate are added to provide the proper environment for anaerobic growth. Hemin and vitamin K1 are growth factors.

### TECHNIQUE

Spread the specimen over the entire agar surface using a sterile loop or swab. Incubate at 36±1°C for 40-48 hours under anaerobic conditions.

### INTERPRETATION of RESULTS

Observe for the growth of colonies.

### STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

### WARNING and PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of ≥1%. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL of WASTE

Disposal of waste must be carried out according to national and local regulations in force.

### REFERENCES

1. Wilkins and Chalgren. (1976). *Antimicrob. Agents Chemother.* **10**: 926.
2. National Committee for Clinical Laboratory Standards. (1993). *Methods for susceptibility testing of anaerobic bacteria. Approved standard M11-A3.* NCCLS, Villanova, Pa.

## PRODUCT SPECIFICATIONS

### NAME

WILKINS CHALGREN AGAR

### PRESENTATION

Ready plates (90 mm) containing 22+/-1 ml of medium.

### STORAGE

10-25 °C

### PACKAGING

Code	Content	Packaging
10053	20 plates	<ul style="list-style-type: none"><li>• 5 plates in thermically soldered film</li><li>• 4 x 5 plates in cardboard boxes</li></ul>
10053*	100 plates	<ul style="list-style-type: none"><li>• 5 plates in thermically soldered film</li><li>• 2 x 5 plates in thermically soldered bag</li><li>• 10 piles (2 x 5 ps) in box</li></ul>

### pH OF THE MEDIUM

7.1 ± 0.1

### USE

WILKINS CHALGREN AGAR is used for susceptibility testing of anaerobes and for isolating and cultivating anaerobes.

### TECHNIQUE

Refer to technical sheet of the product.

### APPEARANCE of the MEDIUM

Light to medium amber medium, slightly opalescent.

### SHELF LIFE

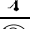
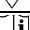
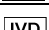
6 months

### QUALITY CONTROL

1. Control of general characteristics, label and print
2. Sterility control  
7 days at 25 ± 1 °C, in aerobiosis  
7 days at 36 ± 1 °C, in aerobiosis
3. Microbiological control  
Inoculum for productivity: 10-100 UFC/ml  
Inoculum for selectivity: 10<sup>4</sup>-10<sup>5</sup> UFC/ml  
Inoculum for specificity: ≤ 10<sup>4</sup> UFC/ml  
Incubation conditions: 40-48 h at 36 ± 1 °C, under anaerobic conditions

Microorganism		Growth
<i>Clostridium perfringens</i>	ATCC 13124	Good

### TABLE of SYMBOLS

Symbol	Meanings
	In vitro Diagnostic Medical Device
	Manufacturer
	Catalogue number
	Use by
	Temperature limitation
	Do not reuse
	Kit content
	Consult accompanying documents
	Batch code

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