CASITONE AGAR

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CASITONE AGAR

Medium for susceptibility testing of yeasts and moulds.

TYPICAL FORMULA (g/l)
Casitone	9.00
Yeast Extract	5.00
Potassium Phosphate Monobasic	0.54
Sodium Phosphate Bibasic	3.34
Sodium Citrate	10.00
Glucose	20.00
Agar	18.00
Final pH 7.0 ± 0.2	

DESCRIPTION

CASITONE AGAR is a medium used for susceptibility testing of yeasts and moulds.

PRINCIPLE

Casitone and yeast extract provide nitrogen, carbon, mineral and vitamins. Potassium and sodium phosphate are the buffer agents. Sodium citrate is a further source of nitrogen and carbon. Glucose is a energy source. Agar is the solidifying agent.

TECHNIQUE

1. Transfer 4-5 colonies in an appropriate broth and incubate at 37°C up to obtain a 0.5 MacFarland suspension.
2. Dip a sterile swab into the suspension and inoculate the plate by streaking the swab onto the entire agar surface.
3. Dispense the discs containing the antimicrobial onto the inoculated agar.
4. Incubate aerobically at 35±2°C for 24-48 hours.

INTERPRETATION OF RESULTS

After incubation measure the inhibition zone diameter for each disc and interpret the values by referring to the current guidelines.

STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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CASITONE AGAR

PRESENTATION

Ready-to-use plates (90 mm) containing 22 ± 1 ml of medium.

STORAGE

10-25°C

PACKAGING

Ref.	Content	Packaging
10079	20 plates	<ul style="list-style-type: none"> • 10 plates in thermally soldered film • 2 x 10 plates in cardboard box

pH OF THE MEDIUM

7.0 ± 0.2

USE

CASITONE AGAR is a medium used for susceptibility testing of yeasts and moulds

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Clear, amber medium

SHELF LIFE

6 months

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at $22 \pm 1^\circ\text{C}$, in aerobiosis
7 days at $35 \pm 1^\circ\text{C}$, in aerobiosis
- Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Incubation Conditions: 24-48 h at $35 \pm 2^\circ\text{C}$, in aerobiosis

Microorganism		Growth
<i>Aspergillus niger</i>	ATCC® 16404	Good
<i>Candida albicans</i>	ATCC® 10231	Good
<i>Saccharomyces cerevisiae</i>	ATCC® 9763	Good

TABLE OF SYMBOLS

LOT	Batch code	IVD	<i>In vitro</i> Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instruction for use		Do not reuse

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net