Bordet Gengou Agar (Sheep blood 15%)

Selective medium for the isolation of *Bordetella pertussis*.

TYPICAL FORMULA (per litre of purified water)

Potato Infusion	4.5 g
Peptic Digest of Animal Tissue	5.0 g
Pancreatic Digest of Casein	5.0 g
Sodium Chloride	5.5 g
Agar	16.0 g
Cefalexin	0.04 g
Glycerol	10 ml
Sheep Blood, defibrinated	150 ml
Final pH 6.7 ± 0.2	

DESCRIPTION

Bordet Gengou Agar (Sheep blood 15%) is a selective medium for the isolation of *Bordetella* spp., including *B. pertussis* and *B. parapertussis*.

PRINCIPLE

Potato infusion, peptic digest of animal tissue and pancreatic digest of casein provide amino acids, nitrogen and vitamins for organisms growth. Glycerol is a source of carbon. Yeast extract is a source of vitamins, particularly of B-group. Sodium chloride maintains the osmotic balance of the medium. Agar is the solidifying agent. Cefalexin partially suppresses normal respiratory tract flora. Sheep blood provides essential growth factors for recovery and cultivation of *Bordetella* spp.

TECHNIQUE

Inoculate the plates as soon as possible after the arrival of the specimens(*) in the laboratory and streak for dilution. Incubate in ambient air in a moist chamber at 35 ± 2°C for 7-10 days, and inspect daily, using a magnifying lens.

(*) Consult appropriate references about specimen collection and types as they may affect significantly the viability of *Bordetella*.

INTERPRETATION OF RESULTS

Colonies of *Bordetella pertussis* appear small, white, domed and glistening, resembling bisected pearls and may be surrounded by a weak beta-hemolysis.

STORAGE

2-8°C away from light, until the expiry date on the label. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is intended for *in vitro* diagnostic use and must be used by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

REFERENCES

1. Isenberg, H.D. Ed. (1992) Clinical microbiology procedures handbook, vol. 1. American Society of Microbiology, Washington, D.C.
2. Woolfrey, B.F., and J.A. Moody (1991) Human infections associated with *Bordetella bronchiseptica*. Clin. Microbiol. Rev. 4: 243-255.
3. Kendrick, P.L., E. Eldering, and W.L. Bradford (1970) Whooping cough. In H.L. Bodily, E.L. Updyke, and J. O. Mason (ed.), Diagnostic procedures for bacterial, mycotic and parasitic infections, 5th ed. American Public Health Association, New York, NY.
4. Bordet, J., and D. Gengou (1906) Le microbe de la coqueluche. Ann. Inst. Pasteur 20: 731.

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Bordet Gengou Agar (Sheep blood 15%)

PRESENTATION

Ready-to-use 90mm plates containing 22 ± 1 ml of medium

STORAGE

2-8°C

PACKAGING

Ref.	Content	Packaging
10353	20 plates	<ul style="list-style-type: none"> • 10 plates in thermally soldered film • 2 x 10 plates in cardboard box
10353*	100 plates	<ul style="list-style-type: none"> • 10 plates in thermally soldered film • 10 x 10 plates in cardboard box

pH OF THE MEDIUM

6.7 ± 0.2

USE

Selective medium for the isolation of *B. pertussis* and *B. parapertussis*

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Opaque, cherry-red

SHELF LIFE

2 months

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
 - 7 days at 22 ± 2°C, in aerobiosis
 - 7 days at 35 ± 2°C, in aerobiosis
- Microbiological control
 - Inoculum for productivity: 50-100 CFU
 - Inoculum for selectivity: 10⁴-10⁶ CFU
 - Incubation Conditions: Aerobic in moist chamber at 35 ± 2°C for 5-7 days

Microorganism

Bordetella bronchiseptica ATCC® 4617
Streptococcus pneumoniae ATCC® 6305
Streptococcus pyogenes ATCC® 19615

Growth

Good
Inhibited
Inhibited

TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
 Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Consult instructions for use	 Do not reuse

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LIOFILCHEM® S.r.l.