MacConkey Agar No.2

Selective and differential medium for detection of enterococci from clinical specimens and environmental samples.

| TYPICAL FORMULA | (g/l) |
|--------------------|-------|
| Peptone | 20.0 |
| Lactose | 10.0 |
| Sodium Chloride | 5.0 |
| Bile Salts No.2 | 1.5 |
| Neutral Red | 0.05 |
| Crystal Violet | 0.001 |
| Agar | 13.5 |
| Final pH 7.2 ± 0.2 | |

DESCRIPTION

MacConkey Agar No.2 is a selective medium used for the identification of enterococci in the presence of non-lactose fermenters and coliform bacteria in water, food and clinical samples.

PRINCIPLE

Peptone provides amino acids, nitrogen, carbon, vitamins and minerals for organisms growth. Lactose is the fermentable carbohydrate. Sodium chloride maintains the osmotic balance of the medium. Bile salts no.2 and crystal violet are the selective agents, inhibiting Gram-positive bacteria. Neutral red is the pH indicator. Agar is the solidifying agent

TECHNIQUE

Inoculate the plates by streaking with the material under investigation to obtain well isolated colonies. Incubate aerobically at 35 ± 2°C for 18-48 hours.

INTERPRETATION OF RESULTS

Enterococci appear as small red colonies. Lactose fermenters cultivate with larger pink colonies. Non-lactose fermenters form colourless colonies.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

REFERENCES

1. McGeachie J. and Kennedy A.C. (1963) J. Clin. Path. 16. 32-38.
2. MacConkey A. (1905) J. Hygiene. 5:333-379.

PRODUCT SPECIFICATIONS

NAME

MacConkey Agar No.2

PRESENTATION

Ready to use plates (90 mm) containing 22±1 ml of medium

STORAGE

2-8°C

PACKAGING

| Ref. | Content | Packaging |
|--------|------------|---|
| 10438 | 20 plates | <ul style="list-style-type: none"> • 10 plates in thermally soldered film • 2 x 10 plates in cardboard box |
| 10438* | 100 plates | <ul style="list-style-type: none"> • 10 plates in thermally soldered film • 10 x 10 plates in cardboard box |

pH OF THE MEDIUM

7.2 ± 0.2

USE

MacConkey Agar No.2 is a selective medium used for the identification of enterococci in the presence of non-lactose fermenters and coliform bacteria in water, food and clinical samples

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Red-violet

SHELF LIFE

6 months

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 22 ± 2°C, in aerobiosis
7 days at 35 ± 2°C, in aerobiosis
- Microbiological control
Inoculum for productivity: 50-100 CFU
Inoculum for selectivity: 10⁴-10⁶ CFU
Incubation Conditions: 18-24 hours at 35 ± 2°C, in aerobiosis

| Microorganism | | Growth | Colony Colour |
|-------------------------------|-------------|-----------|---------------|
| <i>Enterococcus faecalis</i> | ATCC® 29212 | Good | Red |
| <i>Escherichia coli</i> | ATCC® 25922 | Good | Pink |
| <i>Salmonella</i> Enteritidis | ATCC® 13076 | Good | Colourless |
| <i>Staphylococcus aureus</i> | ATCC® 25923 | Inhibited | --- |

TABLE OF SYMBOLS

| | | | | |
|--|---|---|---|---|
|  Batch code |  <i>In vitro</i> Diagnostic Medical Device |  Manufacturer |  Use by |  Fragile, handle with care |
|  Catalogue number |  Temperature limitation |  Contains sufficient for $\langle n \rangle$ tests |  Caution, consult instructions for use |  Do not reuse |

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LIOFILCHEM® S.r.l.