## TKT Edwards Medium Modified

Selective Medium for the isolation of *Streptococcus agalactiae* and other streptococci.

### TYPICAL FORMULA\*

Meat Peptone	10.0 g
Meat Extract	10.0 g
Aesculin	1.0 g
Sodium Chloride	5.0 g
Thallos Sulfate	0.33 g
Crystal Violet	0.0013 g
Blood Defibrinated	50 ml
Staphylococcal Toxin	50 ml
Agar	15.0 g
Final pH 7.4 ± 0.2	

\*Formula may be adjusted and/or supplemented as required to meet performance specifications

### DESCRIPTION

TKT Edwards Medium Modified is a culture medium used for the selective isolation of *S. agalactiae* and other streptococci, which cause bovine mastitis.

### PRINCIPLE

Peptone and meat extract provide amino acids, nitrogen, carbon, vitamins, and other nutrients for microorganisms growth. Sodium chloride maintains the osmotic balance of the medium. Thallos sulfate and crystal violet are the selective agents. Aesculin allows to differentiate aesculin-negative group-B streptococci (blue colonies) from aesculin-positive group-D streptococci (black colonies). The staphylococcal toxin (beta-hemolysin) along with defibrinated blood is incorporated into the medium to observe the CAMP reaction directly on the primary isolation medium. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the plate by spreading the milk sample previously centrifuged. Incubate at 35 ± 2°C for 18-24 h.

### INTERPRETATION OF RESULTS

Observe microbial growth. *S. agalactiae* produces blue colonies.  
Subculture suspect colonies to perform adequate confirmation tests.

### STORAGE

Store at 2-8°C away from light. Do not use the product beyond its expiry date on the label or if product shows any evidence of contamination or any sign of deterioration.

### WARNING AND PRECAUTIONS

**For professional use only.** Operators must be trained and have certain experience in the laboratory methods. Please read the instructions carefully before using this product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.

Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

### REFERENCES

1. Keefe G.P. (1997) Can. Vet. J. 38:429-437.
2. Hauge S. T. and Kohler-Ellingsen J. (1953) Nord. Vet. Med. 5. 539-547.
3. McKenzie D. A. (1941) Vet. Rec. 53. 473-480.
4. Edwards S. J. (1933) J. Comp. Path. Therap. 46 211-217.

**LIOFILCHEM® S.r.l.**

Via Scozia, 64026, Roseto degli Abruzzi (TE) - ITALY  
Tel +39 0858930745 Fax +39 0858930330 www.liofilchem.com liofilchem@liofilchem.com

## PRODUCT SPECIFICATIONS

### NAME

TKT Edwards Medium Modified

### STORAGE

2-8°C

### pH OF THE MEDIUM

7.4 ± 0.2

### USE

TKT Edwards Medium Modified is a solid medium used for the selective isolation of *S. agalactiae* and other streptococci

### SHELF LIFE

90 days

### QUALITY CONTROL

**Appearance of Medium:** Straw yellow with blue tones

#### Expected Cultural Response

Inoculum for productivity: 50-100 CFU

Inoculum for selectivity: 10<sup>4</sup>-10<sup>6</sup> CFU

Incubation: 18-24 h/ 35 ± 2°C

Control strains		Growth	Color	CAMP test
<i>Streptococcus agalactiae</i>	ATCC® 13813	Good	Blue	Positive
<i>Enterococcus faecalis</i>	ATCC® 29212	Good	Blue	Negative
<i>Escherichia coli</i>	ATCC® 25922	Inhibited	---	---
<i>Staphylococcus epidermidis</i>	ATCC® 12228	Inhibited	---	---

### PACKAGING

Ref. 10468      90 mm Plate      20 (2x10) plates

### TABLE OF SYMBOLS

<b>LOT</b> Batch code	 Do not reuse	 Manufacturer	 Use by	 Fragile, handle with care
<b>REF</b> Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult instructions for use	

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