## Campylobacter Skirrow Agar

Selective medium for the isolation of *Campylobacter* spp from clinical specimens.

| TYPICAL FORMULA             | (g/l)   |
|-----------------------------|---------|
| Pancreatic Digest of Casein | 13.0    |
| Heart Infusion              | 3.0     |
| Yeast Extract               | 5.0     |
| Sodium Chloride             | 5.0     |
| Agar                        | 15.0    |
| Lysed Horse Blood           | 70 ml   |
| Polymyxin B                 | 2500 IU |
| Trimethoprim                | 0.005   |
| Vancomycin                  | 0.01    |
| Sodium Pyruvate             | 0.25    |
| Sodium Metabisulphite       | 0.25    |
| Ferrous Sulphate            | 0.25    |
| Final pH 7.3 ± 0.2          |         |

### DESCRIPTION

Campylobacter Skirrow Agar is a selective medium used for the isolation of *Campylobacter* spp from clinical specimens and other materials.

### PRINCIPLE

Pancreatic digest of casein and heart infusion provide amino acids, nitrogen, carbon, vitamins and minerals for organisms growth. Yeast extract is a source of vitamins, particularly of B group. Sodium chloride maintains the osmotic balance of the medium. Agar is the solidifying agent. The selectivity is due to the presence of the antimicrobial agents (polymyxin B, trimethoprim, vancomycin) which suppress the growth of the normal microbial flora in fecal specimens, thereby facilitating isolation of *C. jejuni*. Lysed horse blood supplies the X factor (heme) and other growth requirements. Sodium pyruvate, sodium metabisulphite and ferrous sulphate enhance the recovery of *Campylobacter* spp.

### TECHNIQUE

Inoculate the plates by streaking the sample over the agar surface. Incubate at 42 ± 2°C for 40-48 hours in a reduced oxygen, increased carbon dioxide (microaerobic) atmosphere.

### INTERPRETATION OF RESULTS

*Campylobacter jejuni* produces two types of colonies. One is small, raised, grayish-brown, smooth, glistening with an entire translucent edge. The other colony type is flat, mucoid, grayish, translucent and has an irregular edge. A small percentage of strains may appear tan or slightly pinkish. Colonies tend to spread, especially when initially isolated from fresh clinical specimens.

### STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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Campylobacter Skirrow Agar

### PRESENTATION

Ready to use plates (90 mm) containing  $22 \pm 1$  ml of medium

### STORAGE

2-8°C

### PACKAGING

| Ref.  | Content   | Packaging                                                                                                                      |
|-------|-----------|--------------------------------------------------------------------------------------------------------------------------------|
| 10602 | 20 plates | <ul style="list-style-type: none"> <li>10 plates in thermally soldered film</li> <li>2 x 10 plates in cardboard box</li> </ul> |

### pH OF THE MEDIUM

$7.3 \pm 0.2$

### USE

Campylobacter Skirrow Agar is a selective medium used for the isolation of *Campylobacter* spp from clinical specimens and other materials

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Opaque, cherry red.

### SHELF LIFE

60 days

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control  
7 days at  $22 \pm 2^\circ\text{C}$ , in aerobiosis  
7 days at  $35 \pm 2^\circ\text{C}$ , in aerobiosis
- Microbiological control  
Inoculum for productivity: 50-100 CFU  
Inoculum for selectivity:  $10^4$ - $10^6$  CFU  
Incubation Conditions: 40-48 hours at  $42 \pm 2^\circ\text{C}$  under microaerobic atmosphere

#### Microorganism

*Campylobacter jejuni* ATCC® 33291  
*Escherichia coli* ATCC® 25922

#### Growth

Good  
Inhibited

### TABLE OF SYMBOLS

|            |                  |                                                                                     |                                    |                                                                                     |                                   |                                                                                     |                                       |                                                                                       |                           |
|------------|------------------|-------------------------------------------------------------------------------------|------------------------------------|-------------------------------------------------------------------------------------|-----------------------------------|-------------------------------------------------------------------------------------|---------------------------------------|---------------------------------------------------------------------------------------|---------------------------|
| <b>LOT</b> | Batch code       | <b>IVD</b>                                                                          | In vitro Diagnostic Medical Device |  | Manufacturer                      |  | Use by                                |  | Fragile, handle with care |
| <b>REF</b> | Catalogue number |  | Temperature limitation             |  | Contains sufficient for <n> tests |  | Caution, consult instructions for use |  | Do not reuse              |

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**LIOFILCHEM® S.r.l.**