## CHOCOLATE BACITRACIN AGAR

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## CHOCOLATE BACITRACIN AGAR

Selective medium for the isolation of *Haemophilus* spp.

TYPICAL FORMULA	(g/l)
Proteose Peptone N°3	15.0
Corn Starch	1.0
Potassium Phosphate Dibasic	4.0
Potassium Phosphate Monobasic	1.0
Sodium Chloride	5.0
Horse Blood Defibrinated	50.0 ml
Agar	17.0
Vitalex Growth Supplement	10.0 ml
Bacitracin	0.5
Final pH 7.3 ± 0.2	

### DESCRIPTION

CHOCOLATE BACITRACIN AGAR is a selective medium for the isolation of *Haemophilus* spp.

### PRINCIPLE

Proteose Peptone N°3 represents a source of nitrogen, carbon, sulphur and of other essential growth factors. Potassium phosphate represents the buffering system of the medium. Sodium Chloride maintains the osmotic balance of the medium. Horse sheep defibrinated and Vitalex Growth Supplement provide additional nutrients for the growth of fastidious microorganisms. Bacitracin interferes with the bacterial wall synthesis and it is active mainly on Gram-positive bacteria and on some Gram-negative ones.

### TECHNIQUE

Streak the sample to examine onto the entire surface using a sterile loop. Incubate the plates for 24-48 hours at 36 ± 1 °C, in a hermetically sealed jar containing a 5-10% carbon dioxide environment.

### INTERPRETATION OF RESULTS

*Haemophilus* may appear with small colonies (1 mm diameter), wet, perlaceous, with a typical "rat" smell.

### STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

### REFERENCES

1. Chapin, C.K., G.V. Doern (1983). J. Clin. Microbiol. 17: 1163-1165.
2. Martin, J.E., Armstrong J.H., Smith P.B. (1974). Appl. Microbiol. 27: 802-805.
3. NCCLS document M22-A2, 1996. Quality Assurance for Commercially prepared Microbiological Culture Media-Second ed. Approved Standard.

**LIOFILCHEM® S.r.l.**

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY  
Tel +39 0858930745 Fax +39 0858930330 Website: [www.liofilchem.net](http://www.liofilchem.net) E-mail: [liofilchem@liofilchem.net](mailto:liofilchem@liofilchem.net)

## PRODUCT SPECIFICATIONS

### NAME

CHOCOLATE BACITRACIN AGAR

### PRESENTATION

Ready to use plates (90 mm) containing 22+/-1 ml of medium

### STORAGE

2-8°C

### PACKAGING

Ref.	Content	Packaging
11023	20 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 2 x 10 plates in cardboard box</li> </ul>
10023*	100 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 10 piles (10 x 10 plates) in cardboard box</li> </ul>

### pH OF THE MEDIUM

7.3 ± 0.2

### USE

CHOCOLATE BACITRACIN AGAR is a selective medium for the isolation of *Haemophilus* spp

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Brown opaque medium

### SHELF LIFE

4 months

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
  - 7 days at 22 ± 1°C, in aerobiosis
  - 7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
  - Inoculum for productivity: 10-100 CFU/ml
  - Inoculum for selectivity: 10<sup>4</sup>-10<sup>5</sup> CFU/ml
  - Inoculum for specificity: ≤10<sup>4</sup> CFU/ml
  - Incubation Conditions: 18-24 hours at 36 ± 1°C, in 5-10% CO<sub>2</sub>

Microorganism		Growth
<i>Escherichia coli</i>	ATCC 25922	Good
<i>Staphylococcus aureus</i>	ATCC 25923	Inhibited
<i>Neisseria meningitidis</i>	ATCC 13090	Inhibited
<i>Haemophilus parainfluenzae</i>	ATCC 7901	Good

### TABLE OF SYMBOLS

 LOT	Batch code	 IVD	<i>In vitro</i> Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
 REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instructions for use		Do not reuse

**LIOFILCHEM® S.r.l.**

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY  
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net