## Columbia CNA Agar (Sheep blood 5%)

Selective medium for the isolation of streptococci, staphylococci and pneumococci.

TYPICAL FORMULA	(g/l)
Peptospecial	23.0
Starch	1.0
Sodium Chloride	5.0
Colistin Sulphate	0.01
Nalidixic Acid	0.015
Sheep Blood Defibrinated	50.0 ml
Agar	14.0
Final pH 7.3 ± 0.2	

### DESCRIPTION

Columbia CNA Agar (Sheep blood 5%) is used as a selective growth medium for the isolation and differentiation of Gram-positive cocci from clinical and nonclinical specimens containing mixed flora.

### PRINCIPLE

Peptospecial is a source of nitrogen, sulfur, carbon and other essential growth factors. Sodium chloride maintains osmotic balance of the medium. Colistin and nalidixic acid constitute selective agents: particularly colistin destroys cell membrane of Gram-negative bacteria, whilst nalidixic acid inhibits DNA replication in Gram-negative bacteria. Defibrinated sheep blood supplies further growth factors for fastidious microorganisms and allows to enhance haemolytic reactions. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the plates by streaking the specimen onto the surface of the medium using a sterile loop, in order to isolate single colonies. Incubate at 36 ± 1 °C for 18-48 hours, under aerobic, anaerobic or microaerophilic conditions.

### INTERPRETATION OF RESULTS

Examine plates for growth and hemolytic reactions. Four types of hemolysis on blood agar media can be described:

1.  $\alpha$ -hemolysis is the reduction of hemoglobin to methemoglobin in the medium surrounding the colony, causing a greenish discolouration of the medium.
2.  $\beta$ -hemolysis is the lysis of red blood cells, producing a clear zone surrounding the colony.
3.  $\gamma$ -hemolysis indicates no destruction of red blood cells and no change in the color of the medium.
4.  $\delta$ -hemolysis indicates a partial lysis.

### STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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Columbia CNA Agar (Sheep blood 5%)

### PRESENTATION

Ready to use plates (90 mm) containing 22 ± 1 ml of medium

### STORAGE

2-8°C

### PACKAGING

Ref.	Content	Packaging
11024	20 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 2 x 10 plates in cardboard box</li> </ul>
11024*	100 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 10 piles (10 x 10 plates) in cardboard box</li> </ul>

### pH OF THE MEDIUM

7.3 ± 0.2

### USE

Columbia CNA Agar (Sheep blood 5%) is a selective medium used for the isolation and differentiation of streptococci, staphylococci and pneumococci

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Cherry red, opaque

### SHELF LIFE

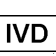
90 days

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
  - 7 days at 22 ± 2°C, in aerobiosis
  - 7 days at 35 ± 2°C, in aerobiosis
- Microbiological control
  - Inoculum for productivity: 50-100 CFU
  - Inoculum for selectivity: 10<sup>4</sup>-10<sup>6</sup> CFU
  - Incubation Conditions: 18-24 hours at 36 ± 1°C

Microorganism		Growth	Haemolysis
<i>Staphylococcus aureus</i>	ATCC® 25923	Good	Beta
<i>Escherichia coli</i>	ATCC® 25922	Inhibited	---
<i>Streptococcus pyogenes</i>	ATCC® 19615	Good	Beta
<i>Proteus mirabilis</i>	ATCC® 25933	Inhibited	---

### TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
 Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult instructions for use	 Do not reuse

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**LIOFILCHEM® S.r.l.**