## TRYPTIC SOY AGAR (Horse blood 5%)

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## TRYPTIC SOY AGAR (Horse blood 5%)

Medium for fastidious bacteria isolation.

TYPICAL FORMULA	(g/l)
Tryptone	15.0
Soytone	5.0
Sodium Chloride	5.0
Horse Blood Defibrinated	50.0 ml
Agar	15.0
Final pH 7.3 ± 0.2	

### DESCRIPTION

TRYPTIC SOY AGAR (Horse blood 5%) is a medium for fastidious bacteria isolation.

### PRINCIPLE

Tryptone and Soytone provide nitrogen, carbon, sulphur and other essential growth factors. Sodium chloride maintains the osmotic balance of the medium. Horse blood defibrinated supplies additional growth factors for fastidious microorganisms and allows to evidence the haemolytic reactions. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the plates by streaking the sample to examine onto the surface of the medium using a sterile loop in order to isolate single colonies. Incubate the plates at 36±1°C for 18-48 hours, in microaerophilic atmosphere.

### INTERPRETATION OF RESULTS

Observe for growth and for haemolytic reactions. Four different kinds of haemolysis can be distinguished:

- alfa-haemolysis: haemoglobin is reduced to metahaemoglobin in the medium surrounding the colony and this causes a greenish decolouring of the medium;
- beta-haemolysis: it is the lysis of erythrocytes which is evident in the bright zone around the colony;
- gamma-haemolysis: any destruction of erythrocytes or any change in the medium does not occur;
- alfa primo-haemolysis: a little zone of complete haemolysis surrounded by a partial lysis area is evident.

### STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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TRYPTIC SOY AGAR (Horse blood 5%)

### PRESENTATION

Ready to use plates (90 mm) containing 22+/-1 ml of medium

### STORAGE

2-8°C

### PACKAGING

Ref.	Content	Packaging
11038	20 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 2 x 10 plates in cardboard box</li> </ul>
11038*	100 plates	<ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 10 piles (10 x 10 plates) in cardboard box</li> </ul>

### pH OF THE MEDIUM

7.3 ± 0.2

### USE

TRYPTIC SOY AGAR ( Sheep blood 5%) is a medium for fastidious bacteria isolation

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Opaque, cherry red medium

### SHELF LIFE

75 days

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
  - 7 days at 22 ± 1°C, in aerobiosis
  - 7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
  - Inoculum for productivity: 10-100 CFU/ml
  - Incubation Conditions: 18-48 hours at 36 ± 1°C

Microorganism		Growth	Haemolysis
<i>Escherichia coli</i>	ATCC 25922	Good	None
<i>Staphylococcus aureus</i>	ATCC 25923	Good	Beta
<i>Streptococcus pneumoniae</i>	ATCC 6305	Good	Alfa
<i>Streptococcus pyogenes</i>	ATCC 19615	Good	Beta

### TABLE OF SYMBOLS

 LOT	Batch code	 IVD	In vitro Diagnostic Medical Device		Manufacturer		Use by		Fragile, handle with care
 REF	Catalogue number		Temperature limitation		Contains sufficient for <n> tests		Caution, consult instructions for use		Do not reuse

**LIOFILCHEM® S.r.l.**

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY  
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net