## COLUMBIA CNA MOD. AGAR (Sheep blood 5%)

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## COLUMBIA CNA MOD. AGAR (Sheep blood 5%)

Selective medium for the streptococci isolation.

| TYPICAL FORMULA          | (g/l)   |
|--------------------------|---------|
| Peptospecial             | 23.0    |
| Starch                   | 1.0     |
| Sodium Chloride          | 5.0     |
| Colistin Sulphate        | 0.01    |
| Nalidixic Acid           | 0.015   |
| Crystal Violet           | 0.002   |
| Defibrinated Sheep Blood | 50.0 ml |
| Agar                     | 14.0    |
| Final pH 7.3 ± 0.2       |         |

### DESCRIPTION

COLUMBIA CNA MOD. AGAR (Sheep blood 5%) is a selective medium for the streptococci isolation.

### PRINCIPLE

Peptospecial is a source of nitrogen, sulfur, carbon and other essential growth factors. Sodium chloride maintains osmotic balance of the medium. Colistin and nalidixic acid constitute selective agents: particularly colistin destroys cell membrane of Gram-negative bacteria, whilst nalidixic acid inhibits DNA replication in Gram-negative bacteria. Crystal violet inhibits the staphylococci growth. Defibrinated sheep blood supplies further growth factors for fastidious microorganisms and allows to enhance haemolytic reactions. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the plates by streaking the specimen onto the surface of the medium using a sterile loop, in order to isolate single colonies. Beside carry out an inoculum in the depth of the medium in order to deposit beta-haemolytic streptococci under agar surface. Incubate the plates at 36±1°C for 18-48- hours, under aerobic, anaerobic or microaerophilic conditions according to the fixed laboratory procedures.

### INTERPRETATION OF RESULTS

Examine plates for growth and hemolytic reactions. Four types of hemolysis on blood agar media can be described:

1.  $\alpha$ -hemolysis is the reduction of hemoglobin to methemoglobin in the medium surrounding the colony, causing a greenish discolorization of the medium.
2.  $\beta$ -hemolysis is the lysis of red blood cells, producing a clear zone surrounding the colony.
3.  $\gamma$ -hemolysis indicates no destruction of red blood cells and no change in the color of the medium.
4.  $\delta$ -hemolysis indicates a partial lysis.

### STORAGE AND TRANSPORT CONDITIONS

6-12°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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COLUMBIA CNA MOD. AGAR(Sheep blood 5%)

### PRESENTATION

Ready to use plates (90 mm) containing 22+/-1 ml of medium

### STORAGE

6-12°C

### PACKAGING

| Ref.   | Content    | Packaging  |
|--------|------------|--|
| 11124  | 20 plates  | <ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 2 x 10 plates in cardboard box</li> </ul>             |
| 11124* | 100 plates | <ul style="list-style-type: none"> <li>• 10 plates in thermally soldered film</li> <li>• 10 piles (10 x 10 plates) in cardboard box</li> </ul> |

### pH OF THE MEDIUM

7.3 ± 0.2

### USE

COLUMBIA CNA MOD. AGAR (Sheep blood 5%) is a selective medium for streptococci, staphylococci and pneumococci isolation

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Cherry red medium, opaque

### SHELF LIFE

75 days

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control  
7 days at 22 ± 1°C, in aerobiosis  
7 days at 36 ± 1°C, in aerobiosis
- Microbiological control  
Inoculum for productivity: 10-100 CFU/ml  
Inoculum for selectivity: 10<sup>4</sup>-10<sup>5</sup> CFU/ml  
Inoculum for specificity: ≤10<sup>4</sup> CFU/ml  
Incubation Conditions: 18-24 hours at 36 ± 1°C

| Microorganism                   |            | Growth    | Haemolysis  |
|---------------------------------|------------|-----------|-------------|
| <i>Streptococcus pyogenes</i>   | ATCC 19615 | Good      | β-hemolysis |
| <i>Streptococcus pneumoniae</i> | ATCC 6303  | Good      | α-hemolysis |
| <i>Proteus mirabilis</i>        | ATCC 12453 | Inhibited | ---         |
| <i>Staphylococcus aureus</i>    | ATCC 25923 | Inhibited | ---         |

### TABLE OF SYMBOLS

|            |                  |   |   |   |                                   |   |                                       |   |                           |
|------------|------------------|---|---|---|-----------------------------------|---|---------------------------------------|---|---------------------------|
| <b>LOT</b> | Batch code       | <b>IVD</b>  | <i>In vitro</i> Diagnostic Medical Device |  | Manufacturer                      |  | Use by                                |  | Fragile, handle with care |
| <b>REF</b> | Catalogue number |  | Temperature limitation                    |  | Contains sufficient for <n> tests |  | Caution, consult instructions for use |  | Do not reuse              |

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**LIOFILCHEM® S.r.l.**