GLUCOSE BROTH

Medium for cultivating fastidious microorganisms and for detecting gas from enteric bacilli.

TYPICAL FORMULA (g/l)

Meat Extract	3.0
Tryptone	10.0
Sodium Chloride	5.0
Glucose	5.0
Final pH 6.9 ± 0.1	

DESCRIPTION

GLUCOSE BROTH is a medium for cultivating fastidious microorganisms and for detecting gas from enteric bacilli.

PRINCIPLE

Meat extract and tryptone are a source of aminoacids and proteins. Glucose is a source of energy and constitutes the substrate of fermentation by glucose fermenting microorganisms. Sodium chloride maintains the osmotic balance of the medium.

PREPARATION

Check the content of the tube is homogeneous and clear; if it is the case repeatedly turn the tube upside down. Glucose Broth can be used as it is or aseptically subdivided into smaller parts.

TECHNIQUE

1. Before use keep the tubes of Glucose Broth at room temperature or at 36+/-1°C.
2. Open the tubes immediately before inoculating and discard all the opened tubes which have not been used.
3. Introduce 1 g or 1 ml of the specimen into the tube.
4. Mix as necessary to obtain a homogeneous suspension.
5. Turn once the tube to let air go out from the Durham tube.
6. Incubate at 36+/-1°C for 24 hours.

INTERPRETATION OF RESULTS

Turbidity in the broth is the sign of bacterial growth. Production of gas is evident in the Durham tube.

STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING and PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of ≥1%. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL of WASTE

Disposal of waste must be carried out according to national and local regulations in force.

REFERENCES

1. APHA (1972) Standard Methods for the Examination of Dairy Products, 13th Ed.
2. APHA (1985) Standard Methods for the Examination of Water and Wastewater, 16th Ed.
3. Flowers, R.S., J. D'Aoust, W.H. Andrews, and J.S. Bailey.(1992). Salmonella, p. 371-442.

Liofilchem s.r.l.

Via Scozia-Zona industriale - 64026 Roseto degli Abruzzi Tel. +39.085.8930745 - Fax +39.085.8930330
Web site: <http://www.liofilchem.net> E-mail: liofilchem@liofilchem.net

PRODUCT SPECIFICATIONS

NAME

GLUCOSE BROTH

PRESENTATION

Glass tubes containing 10 ml of broth.

PACKAGING

Code	Content	Packaging
20105	10 tubes x 10 ml	10 tubes in cardboard box

pH OF THE MEDIUM

6.9 ± 0.1

USE

GLUCOSE BROTH is a medium for cultivating fastidious microorganisms and for detecting gas from enteric bacilli.

TECHNIQUE

Refer to technical sheet of the product.

APPEARANCE OF THE MEDIUM

Light amber, clear without precipitates.

SHELF LIFE

2 years

QUALITY CONTROL

- Control of general characteristics, label and print
- Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Incubation conditions: 24 h at 36 ± 1°C

Microorganism		Growth	Gas
<i>Escherichia coli</i>	ATCC 25922	Good	+
<i>Proteus mirabilis</i>	ATCC 25933	Good	+
<i>Klebsiella pneumoniae</i>	ATCC 13883	Good	+
<i>Pseudomonas aeruginosa</i>	ATCC 27853	Good	-

TABLE OF SYMBOLS

 In vitro Diagnostic Medical Device	 Batch code	 Manufacturer	 Contains sufficient for <n> tests
 Catalogue number	 Temperature limitation	 Use by	 Caution, consult accompanying documents

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