## Haemophilus Test Broth

Liquid medium for quantitative antimicrobial susceptibility testing of *Haemophilus influenzae* as standardized by the CLSI.

TYPICAL FORMULA	(g/l)
Beef Extract	2.0
Acid Hydrolysate of Casein	17.5
Starch	1.5
Yeast Extract	5.0
Hematin	0.015
NAD	0.015
Final pH 7.3 ± 0.1	

### DESCRIPTION

Haemophilus Test Broth is a liquid medium recommended for quantitative antimicrobial susceptibility testing of *Haemophilus influenzae* as standardized by the Clinical and Laboratory Standard Institute (CLSI).

### PRINCIPLE

Acid hydrolysate of casein and beef extract supply amino acids, nitrogen, minerals, vitamins, carbon and other nutrients which support the growth of microorganisms. Starch acts as a protective colloid against toxic molecules which can be present in the medium. Hydrolysis of starch during autoclaving supplies a little amount of glucose, which is a source of energy. Hematin and nicotinamide adenine dinucleotide (NAD) are growth factors necessary for the growth of *Haemophilus* spp.

Broth dilution methods, macrodilution and microdilution, are used to measure quantitatively the in vitro activity of an antimicrobial agent against a given bacterial isolate. To perform the test, a series of tubes/wells is prepared with a broth medium to which various concentrations of the antimicrobial agents are added. Tubes/wells are then inoculated with a standardized suspension of the test organism. After incubation at 35 ± 2°C, the tests are examined and the MIC is determined.

### TECHNIQUE

1. Prepare antimicrobial agents in serial two-fold dilutions in Haemophilus Test Broth.
2. Use direct colony suspension procedure to prepare a standardized suspension of the test organism in Mueller Hinton Broth or saline from an overnight Chocolate Agar plate (ref. 10023).
3. Add the suitable volume of the adjusted inoculum to each tube containing the antimicrobial agent in the dilution series, and mix.
4. Incubate tubes aerobically at 35 ± 2°C for 20-24 hours.

### INTERPRETATION OF RESULTS

After incubation the presence of turbidity indicates growth of the organism. The lowest concentration of antimicrobial agent showing no growth is the MIC of that organism for that agent. Interpret the MIC by referring to the current CLSI document M100 and report the organism as susceptible, intermediate or resistant to the agents that have been tested.

### STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for professional use only and must be used by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

### REFERENCES

- Bauer, Kirby, Sherris and Turck. 1966. Am. J. Clin. Pathol. 45:493.
- Ryan, Schoenknecht and Kirby. 1970. Hospital Practice 5:91.
- Barry, Garcia and Thrupp. 1970. Am. J. Clin. Pathol. 53:149.
- CLSI. Methods for dilution Antimicrobial Susceptibility Tests for Bacteria That Grow Aerobically; Approved Standard - Eighth Edition. CLSI document M07-A8. Wayne, PA: Clinical and Laboratory Standards Institute; 2009.

## PRODUCT SPECIFICATIONS

### NAME

Haemophilus Test Broth

### PRESENTATION

Glass tubes containing 10 ml of medium

### STORAGE

10-25°C

### PACKAGING

Ref.	Content	Packaging
24091	20 x 10 ml tubes	20 tubes in cardboard box

### pH OF THE MEDIUM

7.3 ± 0.1

### USE

Haemophilus Test Broth is a liquid medium recommended for quantitative antimicrobial susceptibility testing of *Haemophilus influenzae* as standardized by the Clinical and Laboratory Standard Institute (CLSI)

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Light amber, clear

### SHELF LIFE

1 year

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control  
7 days at 22 ± 2°C, in aerobiosis  
7 days at 35 ± 2°C, in aerobiosis
- Microbiological control  
Inoculum: ≤ 5 x 10<sup>5</sup> CFU/ml  
Incubation Conditions: 35 ± 2°C for 20-24 h

Microorganism		Growth
<i>Haemophilus influenzae</i>	ATCC® 49247	Good
<i>Haemophilus influenzae</i>	ATCC® 49766	Good
<i>Escherichia coli</i>	ATCC® 35218	Good

### TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
 Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult instruction for use	 Do not reuse

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**LIOFILCHEM® S.r.l.**