TODD HEWITT BROTH w Colistin/Nalidixic Acid

Selective medium for cultivating and isolating group B streptococci.

TYPICAL FORMULA	(g/l)
Enzymatic Digest of Casein	20.0
Heart Extract	10.0
Dextrose	2.0
Sodium Chloride	2.0
Disodium Phosphate	0.4
Sodium Carbonate	2.0
Colistin Sulfate	0.01
Nalidixic Acid	0.015
Final pH 7.8 ± 0.2	

DESCRIPTION

TODD HEWITT BROTH w Colistin/Nalidixic Acid is a selective medium used for cultivating and isolating group B streptococci (*Streptococcus agalactiae*).

PRINCIPLE

Enzymatic digest of casein and heart extract are sources of carbon, nitrogen, vitamins and minerals providing an excellent nutritional base for the growth of streptococci. Dextrose stimulates hemolysin production. Disodium phosphate and sodium carbonate provide buffering action to counteract the acidity produced during the fermentation of the carbohydrate, thereby protecting the hemolysin from inactivation by the acid. Colistin sulfate and nalidixic acid suppress growth of gram-negative bacteria.

TECHNIQUE

Inoculate TODD HEWITT BROTH w Colistin/Nalidixic Acid tubes with swab specimens. Incubate at 37°C for 18-24 hours in an aerobic atmosphere. Refer to appropriate texts for further information regarding procedures.

INTERPRETATION OF RESULTS

The turbidity in the tube indicates microbial growth. If turbidity is observed, subculture from the broth culture to a sheep blood agar plate; otherwise, incubate an additional 24 hours before discarding.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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TODD HEWITT BROTH w Colistin/Nalidixic Acid

PRESENTATION

Glass tubes containing 10 ml of medium

STORAGE

2-8°C

PACKAGING

Ref.	Content	Packaging
24145	20 tubes x 10 ml	20 tubes in cardboard box

pH OF THE MEDIUM

7.8 ± 0.2

USE

TODD HEWITT BROTH w Colistin/Nalidixic Acid is a selective medium used for cultivating and isolating group B streptococci (*Streptococcus agalactiae*)

TECHNIQUE

Refer to technical sheet of the product

APPEARANCE OF THE MEDIUM

Amber medium, clear without precipitates

SHELF LIFE

1 year

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 22 ± 1°C, in aerobiosis
7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
Inoculum for productivity: 10-100 UFC/ml
Inoculum for selectivity: 10⁴-10⁵ UFC/ml
Incubation Conditions: 18-24 h at 35 ± 2°C, in aerobiosis

Microorganisms

Microorganisms	ATCC	Growth
<i>Streptococcus agalactiae</i>	ATCC 12386	Good
<i>Enterococcus faecalis</i>	ATCC 51299	Good
<i>Escherichia coli</i>	ATCC 25922	Inhibition

TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
 Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult instructions for use	 Do not reuse

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LIOFILCHEM® S.r.l.