## Motility Test Medium

Semisolid medium for detection of motility of enteric organisms.

TYPICAL FORMULA	(g/l)
Enzymatic Digest of Casein	10.0
Sodium Chloride	5.0
Agar	5.0
Final pH 7.2 ± 0.2	

### DESCRIPTION

Motility Test Medium is a semisolid medium used for the detection of bacterial motility.

### PRINCIPLE

Enzymatic digest of casein provide amino acids, carbon, nitrogen, vitamins and minerals for organisms growth. Sodium chloride maintains the osmotic balance of the medium. Agar is the solidifying agent.

### TECHNIQUE

Inoculate the medium with a pure culture by stabbing the center of the tube to about one-half its length. Incubate tubes with loosened caps at 35 ± 2°C for 18-48 hours in an aerobic atmosphere.

### INTERPRETATION OF RESULTS

Examine tubes for growth and presence of motility. Motility is manifested macroscopically by a diffuse zone of growth spreading from the line of inoculation. Certain species of motile bacteria show diffuse growth throughout the entire medium, while others may exhibit diffusion from one or two points only, appearing as nodular growths along the line of the stab.

### STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

### WARNING AND PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. The product is designed for professional use only and must be used by properly trained operators.

### DISPOSAL OF WASTE

Disposal of waste must be carried out according to the national and local regulations in force.

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Motility Test Medium

### PRESENTATION

Glass tubes containing 10 ml of medium

### STORAGE

10-25°C

### PACKAGING

Ref.	Content	Packaging
24342	20 x 10 ml tubes	20 tubes in cardboard box

### pH OF THE MEDIUM

7.2 ± 0.2

### USE

Motility Test Medium is a semisolid medium used for the detection of bacterial motility

### TECHNIQUE

Refer to technical sheet of the product

### APPEARANCE OF THE MEDIUM

Clear to slightly hazy, yellowish

### SHELF LIFE

2 years

### QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control  
7 days at 22 ± 2°C, in aerobiosis  
7 days at 35 ± 2°C, in aerobiosis
- Microbiological control  
Inoculum for productivity: 10<sup>-1</sup> dilution of fresh cultures in Tryptic Soy Broth  
Incubation Conditions: 35 ± 2°C for 24-48 h

Microorganism		Growth	Motility
<i>Escherichia coli</i>	ATCC® 25922	Good	+
<i>Salmonella Typhimurium</i>	ATCC® 14028	Good	+
<i>Shigella flexneri</i>	ATCC® 12022	Good	-

### TABLE OF SYMBOLS

 <b>LOT</b> Batch code	 <b>IVD</b> <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Use by	 Fragile, handle with care
 <b>REF</b> Catalogue number	 Temperature limitation	 Contains sufficient for <n> tests	 Caution, consult instruction for use	 Do not reuse

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**LIOFILCHEM® S.r.l.**