MUELLER HINTON AGAR

Medium for the growth of non-fastidious microorganisms.

TYPICAL FORMULA (g/L)

Meat Extract	2.0
Casamino Acids, Technical	17.5
Starch	1.5
Agar	15.0
Final pH 7.3 ± 0.1	

DESCRIPTION

MUELLER HINTON AGAR is used for the growth of non-fastidious microorganisms.

PRINCIPLE

Casamino acids and meat extract are a source of amino acids, nitrogen, minerals, vitamins, carbon and other factors which increase the growth of microorganisms. Starch acts as a protective substance against toxic molecules which can be present in the medium. Hydrolysis of starch during sterilization supplies a little amount of glucose which represents a source of energy. Agar is the solidifying agent.

TECHNIQUE

Inoculate the medium with the specimen by stabbing the butt and streaking the slope using a sterile loop to ensure an adequate dispersion of the organisms. Tubes are incubated at 36±1°C for 24-96 hours.

INTERPRETATION OF RESULTS

Observe for microorganisms growth on the medium.

STORAGE

10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

WARNING and PRECAUTIONS

The product is not classified as hazardous by current legislation and does not contain harmful substances in concentrations of ≥1%. The product is designed for *In vitro* diagnostic use and must be used only by properly trained operators.

DISPOSAL of WASTE

Disposal of waste must be carried out according to national and local regulations in force.

REFERENCES

1. Bauer et al. (1966). J. Clin. Pathol. 45:493-496.
2. Mueller, J.H., and Hinton. 1941. Proc. Soc. Exp. Biol. Med. 48: 330-333.
3. NCCLS. Performance standards for susceptibility testing; Twelve Informational Supplement. NCCLS Document M100-S12, January 2002.

Liofilchem s.r.l.

Via Scozia-Zona industriale - 64026 Roseto degli Abruzzi Tel. +39.085.8930745 - Fax +39.085.8930330
Web site: <http://www.liofilchem.net> E-mail: liofilchem@liofilchem.net

PRODUCT SPECIFICATIONS

NAME

MUELLER HINTON AGAR

PRESENTATION

Ready-to-use glass tubes containing 14+/-1 ml.

STORAGE

10-25°C

PACKAGING

Code	Content	Packaging
30090	10 tubes x 10 ml	10 tubes in cardboard box

pH OF THE MEDIUM

7.3 ± 0.1

USE

MUELLER HINTON AGAR is used for the growth of non-fastidious microorganisms.

TECHNIQUE

Refer to technical sheet of the product.

APPEARANCE OF THE MEDIUM

Amber medium, slightly opalescent.

SHELF LIFE

1 year

QUALITY CONTROL

- Control of general characteristics, label and print
- Sterility control
7 days at 25 ± 1°C, in aerobiosis
7 days at 36 ± 1°C, in aerobiosis
- Microbiological control
Incubation conditions: 18-24 h at 36 ± 1°C

Microorganism		Growth	Characteristics
<i>Enterococcus faecalis</i>	ATCC 29212	Good	White colonies
<i>Escherichia coli</i>	ATCC 25922	Good	Colorless colonies
<i>Proteus mirabilis</i>	ATCC 25933	Good	Colorless colonies
<i>Staphylococcus aureus</i>	ATCC 25923	Good	White colonies

TABLE of SYMBOLS

IVD In vitro Diagnostic Medical Device	LOT Batch code	 Manufacturer	 Contains sufficient for <n> tests
REF Catalogue number	 Temperature limitation	 Use by	 Caution, consult accompanying documents

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