

MIC Test Strip

for antimicrobial susceptibility testing

Quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.)


MIC Test Strip is a quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.) of antimicrobial agents against microorganisms and for detecting the resistance mechanisms.

MIC Test Strip are porous strips with special features (**International Patent**) that are impregnated with a predefined concentration gradient of antibiotic, across 15 two-fold dilutions of a conventional M.I.C. method.

On one side of the strip is indicated a M.I.C. scale in $\mu\text{g/mL}$ and a code that identify the antimicrobial agent.

For ESBL, MBL, GRD, AmpC and KPC detection, the double-sided gradient carries the appropriate diagnostic reagents.


MIC Test Strip are available in a large variety of configurations. Each configuration is available in packages of 10, 30 and 100 tests.

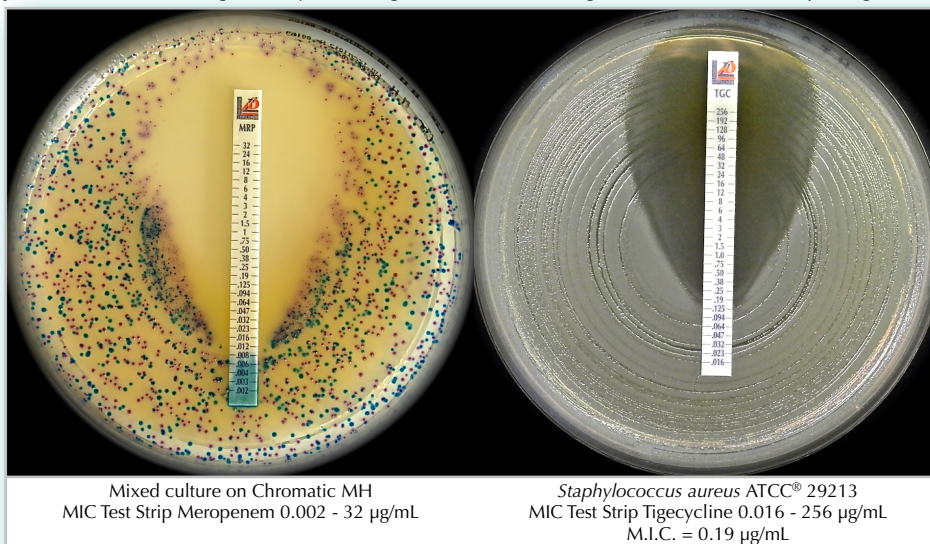

method principle

When the **MIC Test Strip** is applied onto an inoculated agar surface, the preformed exponential gradient of antimicrobial agent is immediately transferred to the agar matrix. After 18 hours incubation or longer, a symmetrical inhibition ellipse centered along the strip is formed. The MIC is read directly from the scale in terms of $\mu\text{g/mL}$ at the point where the edge of the inhibition ellipse intersects the strip **MIC Test Strip**.

Other growth/inhibition patterns may also be seen for resistance detection methods.

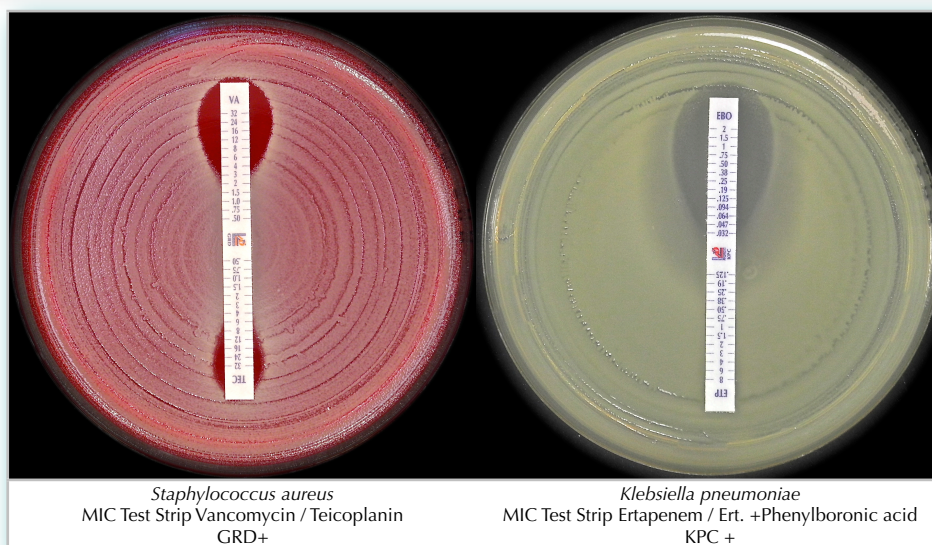

BIBLIOGRAPHY

- CLSI M100-S27, 2017. Performance Standards for Antimicrobial Susceptibility Testing.
- CLSI M7-A10, 2015. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that Grow Aerobically.
- CLSI M11-A8 Methods for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M11-S1 Performance Standards for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M27-A3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Approved Standard - Third Edition.
- CLSI M27-S4. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Fourth Informational supplement.
- EUCAST. Breakpoint tables for interpretation of MICs and zone diameters Version 7.1, March 2017.

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMIKACIN	0.016 - 256	AK	10 strips 30 strips 100 strips	920181 92018 920180
MIC Test Strip	AMOXICILLIN	0.016 - 256	AML	10 strips 30 strips 100 strips	920211 92021 920210
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2/1) <i>CLSI recommended</i>	0.016 - 256*	AUG	10 strips 30 strips 100 strips	920241 92024 920240
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	AMC	10 strips 30 strips 100 strips	921801 92180 921800
MIC Test Strip	AMPICILLIN	0.016 - 256	AMP	10 strips 30 strips 100 strips	920031 92003 920030
MIC Test Strip	AMPICILLIN*-SULBACTAM (2/1) <i>CLSI recommended</i>	0.016 - 256*	AMS	10 strips 30 strips 100 strips	920271 92027 920270
MIC Test Strip	AMPICILLIN*-SULBACTAM (4 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	SAM	10 strips 30 strips 100 strips	921811 92181 921810
MIC Test Strip	AZITHROMYCIN	0.016 - 256	AZM	10 strips 30 strips 100 strips	920301 92030 920300
MIC Test Strip	AZTREONAM	0.016 - 256	ATM	10 strips 30 strips 100 strips	920331 92033 920330
MIC Test Strip	AZTREONAM	0.064 - 1024	ATM	10 strips 30 strips 100 strips	921731 92173 921730
MIC Test Strip	BACITRACIN	0.016 - 256	BA	10 strips 30 strips 100 strips	920191 92019 920190
MIC Test Strip	CEFACLOR	0.016 - 256	CEC	10 strips 30 strips 100 strips	920361 92036 920360
MIC Test Strip	CEFAZOLIN	0.016 - 256	KZ	10 strips 30 strips 100 strips	921741 92174 921740
MIC Test Strip	CEFEPIME	0.016 - 256	FEP	10 strips 30 strips 100 strips	921261 92126 921260
MIC Test Strip	CEFIXIME	0.016 - 256	CFM	10 strips 30 strips 100 strips	920601 92060 920600
MIC Test Strip	CEFOPERAZONE*-SULBACTAM (2/1)	0.016 - 256*	CPS	10 strips 30 strips 100 strips	920231 92023 920230
MIC Test Strip	CEFOTAXIME	0.002 - 32	CTX	10 strips 30 strips 100 strips	920071 92007 920070
MIC Test Strip	CEFOTAXIME	0.016 - 256	CTX	10 strips 30 strips 100 strips	920061 92006 920060
MIC Test Strip	CEFOTETAN	0.016 - 256	CTT	10 strips 30 strips 100 strips	920201 92020 920200
MIC Test Strip	CEFOXITIN	0.016 - 256	FOX	10 strips 30 strips 100 strips	920661 92066 920660
MIC Test Strip	CEFPIROME	0.016 - 256	CR	10 strips 30 strips 100 strips	920081 92008 920080
MIC Test Strip	CEFPODOXIME	0.016 - 256	PX	10 strips 30 strips 100 strips	920051 92005 920050
MIC Test Strip	CEFTAROLINE	0.002 - 32	CPT	10 strips 30 strips 100 strips	920561 92056 920560
MIC Test Strip	CEFTAROLINE	0.016 - 256	CPT	10 strips 30 strips 100 strips	920491 92049 920490
MIC Test Strip	CEFTAZIDIME (FDA cleared)	0.016 - 256	CAZ	10 strips 30 strips 100 strips	921381 92138 921380
MIC Test Strip	CEFTAZIDIME*-AVIBACTAM	0.016 - 256*	CZA	10 strips 30 strips 100 strips	921391 92139 921390
MIC Test Strip	CEFTIBUTEN	0.002 - 32	CTB	10 strips 30 strips 100 strips	920581 92058 920580
MIC Test Strip	CEFTIZOXIME	0.016 - 256	CZX	10 strips 30 strips 100 strips	920161 92016 920160
MIC Test Strip	CEFTOBIPROLE	0.002 - 32	BPR	10 strips 30 strips 100 strips	921401 92140 921400
MIC Test Strip	CEFTOLOZANE*-TAZOBACTAM (FDA cleared)	0.016 - 256*	C/T	10 strips 30 strips 100 strips	921461 92146 921460

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFTRIAXONE	0.016 - 256	CRO	10 strips	920421
				30 strips	92042
				100 strips	920420
MIC Test Strip	CEFTRIAXONE	0.002 - 32	CRO	10 strips	920431
				30 strips	92043
				100 strips	920430
MIC Test Strip	CEFUROXIME	0.016 - 256	CXM	10 strips	921291
				30 strips	92129
				100 strips	921290
MIC Test Strip	CEPHALOTHIN	0.016 - 256	KF	10 strips	920391
				30 strips	92039
				100 strips	920390
MIC Test Strip	CHLORAMPHENICOL	0.016 - 256	C	10 strips	920751
				30 strips	92075
				100 strips	920750
MIC Test Strip	CIPROFLOXACIN	0.002 - 32	CIP	10 strips	920451
				30 strips	92045
				100 strips	920450
MIC Test Strip	CLARITHROMYCIN	0.016 - 256	CLR	10 strips	920481
				30 strips	92048
				100 strips	920480
MIC Test Strip	CLINDAMYCIN (FDA cleared)	0.016 - 256	CD	10 strips	920721
				30 strips	92072
				100 strips	920720
MIC Test Strip	CLOXACILLIN	0.016 - 256	CX	10 strips	920441
				30 strips	92044
				100 strips	920440
MIC Test Strip	COLISTIN	0.016 - 256	CS	10 strips	921411
				30 strips	92141
				100 strips	921410
MIC Test Strip	COLISTIN	0.064 - 1024	CS	10 strips	921421
				30 strips	92142
				100 strips	921420
MIC Test Strip	DALBAVANCIN (FDA cleared)	0.002 - 32	DAL	10 strips	921371
				30 strips	92137
				100 strips	921370
MIC Test Strip	DAPTOMYCIN (Includes Ca ²⁺)	0.016 - 256	DAP	10 strips	921451
				30 strips	92145
				100 strips	921450
MIC Test Strip	DELAFLORACIN (FDA cleared)	0.002 - 32	DLX	10 strips	920801
				30 strips	92080
				100 strips	920800
MIC Test Strip	DORIPENEM	0.002 - 32	DOR	10 strips	920401
				30 strips	92040
				100 strips	920400
MIC Test Strip	DOXYCYCLINE	0.016 - 256	DXT	10 strips	921561
				30 strips	92156
				100 strips	921560
MIC Test Strip	ENROFLOXACIN	0.002 - 32	ENR	10 strips	920131
				30 strips	92013
				100 strips	920130
MIC Test Strip	ERTAPENEM	0.002 - 32	ETP	10 strips	921571
				30 strips	92157
				100 strips	921570
MIC Test Strip	ERYTHROMYCIN (FDA cleared)	0.016 - 256	E	10 strips	920511
				30 strips	92051
				100 strips	920510
MIC Test Strip	FLORFENICOL	0.016 - 256	FFC	10 strips	920611
				30 strips	92061
				100 strips	920610
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.016 - 256	FOS	10 strips	920781
				30 strips	92078
				100 strips	920780
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.064 - 1024	FOS	10 strips	920791
				30 strips	92079
				100 strips	920790
MIC Test Strip	FUSIDIC ACID	0.016 - 256	FU	10 strips	920021
				30 strips	92002
				100 strips	920020
MIC Test Strip	GATIFLOXACIN	0.002 - 32	GAT	10 strips	920111
				30 strips	92011
				100 strips	920110
MIC Test Strip	GEMIFLOXACIN	0.002 - 32	GEM	10 strips	920351
				30 strips	92035
				100 strips	920350
MIC Test Strip	GENTAMICIN	0.016 - 256	CN	10 strips	920091
				30 strips	92009
				100 strips	920090
MIC Test Strip	GENTAMICIN	0.064 - 1024	CN	10 strips	920101
				30 strips	92010
				100 strips	920100
MIC Test Strip	IMIPENEM	0.002 - 32	IMI	10 strips	920541
				30 strips	92054
				100 strips	920540
MIC Test Strip	KANAMYCIN	0.016 - 256	K	10 strips	920341
				30 strips	92034
				100 strips	920340
MIC Test Strip	LEVOFLOXACIN	0.002 - 32	LEV	10 strips	920811
				30 strips	92081
				100 strips	920810

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	LINEZOLID	0.016 - 256	LNZ	10 strips 30 strips 100 strips	921351 92135 921350
MIC Test Strip	MARBOFLOXACIN	0.002 - 32	MAR	10 strips 30 strips 100 strips	920621 92062 920620
MIC Test Strip	MECILLINAM	0.016 - 256	MEC	10 strips 30 strips 100 strips	920171 92017 920170
MIC Test Strip	MEROPENEM (FDA cleared)	0.002 - 32	MRP	10 strips 30 strips 100 strips	920841 92084 920840
MIC Test Strip	MEROPENEM	0.016 - 256	MRP	10 strips 30 strips 100 strips	920851 92085 920850
MIC Test Strip	MEROPENEM*-VABORBACTAM (8 µg/mL) (FDA cleared)	0.016 - 256*	M/V	10 strips 30 strips 100 strips	920741 92074 920740
MIC Test Strip	METRONIDAZOLE	0.016 - 256	MTZ	10 strips 30 strips 100 strips	920871 92087 920870
MIC Test Strip	MOXIFLOXACIN	0.002 - 32	MXF	10 strips 30 strips 100 strips	920901 92090 920900
MIC Test Strip	MUPIROCIN	0.064 - 1024	MUP	10 strips 30 strips 100 strips	920381 92038 920380
MIC Test Strip	MINOCYCLINE	0.016 - 256	MN	10 strips 30 strips 100 strips	920321 92032 920320
MIC Test Strip	NALIDIXIC ACID	0.016 - 256	NA	10 strips 30 strips 100 strips	921321 92132 921320
MIC Test Strip	NETILMICIN	0.016 - 256	NET	10 strips 30 strips 100 strips	920931 92093 920930
MIC Test Strip	NORFLOXACIN	0.016 - 256	NOR	10 strips 30 strips 100 strips	920961 92096 920960
MIC Test Strip	NITROFURANTOIN	0.032 - 512	F	10 strips 30 strips 100 strips	920221 92022 920220
MIC Test Strip	OFLOXACIN	0.002 - 32	OFX	10 strips 30 strips 100 strips	920991 92099 920990
MIC Test Strip	OXACILLIN	0.016 - 256	OX	10 strips 30 strips 100 strips	920151 92015 920150
MIC Test Strip	PENICILLIN G	0.002 - 32	P	10 strips 30 strips 100 strips	921031 92103 921030
MIC Test Strip	PENICILLIN G	0.016 - 256	P	10 strips 30 strips 100 strips	921021 92102 921020
MIC Test Strip	PIPERACILLIN	0.016 - 256	PIP	10 strips 30 strips 100 strips	921051 92105 921050
MIC Test Strip	PIPERACILLIN*-TAZOBACTAM (4 µg/mL)	0.016 - 256*	TZP	10 strips 30 strips 100 strips	921081 92108 921080
MIC Test Strip	POLYMYXIN B	0.064 - 1024	PB	10 strips 30 strips 100 strips	920041 92004 920040
MIC Test Strip	QUINUPRISTIN-DALFOPRISTIN	0.002 - 32	QDA	10 strips 30 strips 100 strips	920261 92026 920260
MIC Test Strip	RIFAMPICIN	0.002 - 32	RD	10 strips 30 strips 100 strips	920011 92001 920010
MIC Test Strip	RIFAMPICIN	0.016 - 256	RD	10 strips 30 strips 100 strips	920251 92025 920250
MIC Test Strip	SPECTINOMYCIN	0.064 - 1024	SPC	10 strips 30 strips 100 strips	920141 92014 920140
MIC Test Strip	SPIRAMYCIN	0.002 - 32	SP	10 strips 30 strips 100 strips	920461 92046 920460
MIC Test Strip	STREPTOMYCIN	0.064 - 1024	S	10 strips 30 strips 100 strips	921111 92111 921110
MIC Test Strip	SULBACTAM	0.016 - 256	SUL	10 strips 30 strips 100 strips	920281 92028 920280
MIC Test Strip	SULFAMETHOXAZOLE	0.064 - 1024	SMX	10 strips 30 strips 100 strips	920311 92031 920310
MIC Test Strip	TEDIZOLID (FDA cleared)	0.002 - 32	TZD	10 strips 30 strips 100 strips	921361 92136 921360

MIC Test Strip for ANTIMICROBIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	TEICoplanin	0.016 - 256	TEC	10 strips	920121
				30 strips	92012
				100 strips	920120
MIC Test Strip	TELAVANCIN	0.002 - 32	TLV	10 strips	920521
				30 strips	92052
				100 strips	920520
MIC Test Strip	TELAVANCIN (FDA cleared)	0.016 - 256	TLV	10 strips	920531
				30 strips	92053
				100 strips	920530
MIC Test Strip	TEMOCILLIN	0.064 - 1024	TMO	10 strips	920291
				30 strips	92029
				100 strips	920290
MIC Test Strip	TETRACYCLINE	0.016 - 256	TE	10 strips	921141
				30 strips	92114
				100 strips	921140
MIC Test Strip	TIAMULIN	0.002 - 32	TIA	10 strips	922001
				30 strips	92200
				100 strips	922000
MIC Test Strip	TICARCILLIN*-CLAVULANIC ACID (2 µg/mL)	0.016 - 256*	TTC	10 strips	921171
				30 strips	92117
				100 strips	921170
MIC Test Strip	TIGECYCLINE	0.016 - 256	TGC	10 strips	921441
				30 strips	92144
				100 strips	921440
MIC Test Strip	TILMICOSIN	0.002 - 32	TIL	10 strips	922011
				30 strips	92201
				100 strips	922010
MIC Test Strip	TOBRAMYCIN	0.016 - 256	TOB	10 strips	921211
				30 strips	92121
				100 strips	921210
MIC Test Strip	TOBRAMYCIN	0.064 - 1024	TOB	10 strips	921201
				30 strips	92120
				100 strips	921200
MIC Test Strip	TRIMETHOPRIM	0.002 - 32	TM	10 strips	920371
				30 strips	92037
				100 strips	920370
MIC Test Strip	TRIMETHOPRIM*-SULFAMETHOXAZOLE (1/19)	0.002 - 32*	SXT	10 strips	921231
				30 strips	92123
				100 strips	921230
MIC Test Strip	VANCOMYCIN (FDA cleared)	0.016 - 256	VA	10 strips	920571
				30 strips	92057
				100 strips	920570

MIC Test Strip for ANTIFUNGAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMPHOTERICIN B	0.002 - 32	AMB	10 strips	921531
				30 strips	92153
				100 strips	921530
MIC Test Strip	ANIDULAFUNGIN	0.002 - 32	AND	10 strips	921551
				30 strips	92155
				100 strips	921550
MIC Test Strip	CASPOFUNGIN	0.002 - 32	CAS	10 strips	921541
				30 strips	92154
				100 strips	921540
MIC Test Strip	FLUCONAZOLE	0.016 - 256	FLU	10 strips	921471
				30 strips	92147
				100 strips	921470
MIC Test Strip	FLUCYTOSINE	0.002 - 32	FC	10 strips	921491
				30 strips	92149
				100 strips	921490
MIC Test Strip	ISAVUCONAZOLE	0.002 - 32	IVU	10 strips	921841
				30 strips	92184
				100 strips	921840
MIC Test Strip	ITRACONAZOLE	0.002 - 32	ITC	10 strips	921481
				30 strips	92148
				100 strips	921480
MIC Test Strip	KETOCONAZOLE	0.002 - 32	KE	10 strips	921511
				30 strips	92151
				100 strips	921510
MIC Test Strip	MICA FUNGIN	0.002 - 32	MYC	10 strips	921821
				30 strips	92182
				100 strips	921820
MIC Test Strip	POSACONAZOLE	0.002 - 32	POS	10 strips	921521
				30 strips	92152
				100 strips	921520
MIC Test Strip	VORICONAZOLE	0.002 - 32	VO	10 strips	921501
				30 strips	92150
				100 strips	921500

MIC Test Strip for KPC detection					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MEROPENEM / MEROPENEM + PHENYLBORONIC ACID	0.125-8 / 0.032-2	MRP/MBO	10 strips	921671
				30 strips	92167
				100 strips	921670
MIC Test Strip	ERTAPENEM / ERTAPENEM + PHENYLBORONIC ACID	0.125-8 / 0.032-2	ETP/EBO	10 strips	921681
				30 strips	92168
				100 strips	921680

MIC Test Strip ESBL for confirmation of Extended Spectrum Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFEPIME/CEFEPIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.064-4	FEP/FEL	10 strips	921611
				30 strips	92161
				100 strips	921610
MIC Test Strip	CEFOTAXIME/CEFOTAXIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.016-1	CTX/CTL	10 strips	921601
				30 strips	92160
				100 strips	921600
MIC Test Strip	CEFTAZIDIME/CEFTAZIDIME+CLAVULANIC ACID (4 µg/mL)	0.5-32 / 0.064-4	CAZ/CAL	10 strips	921591
				30 strips	92159
				100 strips	921590

MIC Test Strip MBL for detection of Metallo Beta-Lactamase					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	4-256 / 1-64	IMI/IMD	10 strips	921621
				30 strips	92162
				100 strips	921620
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	0.125-8 / 0.032-2	IMI/IMD	10 strips	921661
				30 strips	92166
				100 strips	921660
MIC Test Strip	MEROPENEM / MEROPENEM + EDTA	0.125-8 / 0.032-2	MRP/MRD	10 strips	921651
				30 strips	92165
				100 strips	921650

MIC Test Strip GRD for detection of Glycopeptide Resistance					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	VANCOMYCIN / TEICoplanin	0.5-32 / 0.5-32	VA/TEC	10 strips	921631
				30 strips	92163
				100 strips	921630

MIC Test Strip for AmpC detection					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFOTETAN / CEFOTETAN+CLOXACILLIN	0.5-32 / 0.5-32	CTT/CXT	10 strips	921641
				30 strips	92164
				100 strips	921640
MIC Test Strip	ERTAPENEM / ERTAPENEM + CLOXACILLIN	0.125-8 / 0.032-2	ETP/ECX	10 strips	921691
				30 strips	92169
				100 strips	921690




MIC Test Strip for ANTIMYCOBACTERIAL susceptibility testing					
Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	ETHAMBUTOL	0.016 - 256	EB	10 strips	921701
				30 strips	92170
				100 strips	921700
MIC Test Strip	ETHIONAMIDE	0.016 - 256	ET	10 strips	921721
				30 strips	92172
				100 strips	921720
MIC Test Strip	ISONIAZIDE	0.016 - 256	IZ	10 strips	921711
				30 strips	92171
				100 strips	921710

MTS Synergy Application System (International Patent)		
Description	Packaging	Ref.
MTS Synergy Applicator Platform	1 unit	96860
MTS Synergy Delivery Tool	10 disposable applicators	96870

Accessories		
Description	Packaging	Ref.
Rotating Plate Inoculator for confluent microbial growth on the 90 mm petri dish		96794
DEN-1B McFarland densitometer with built-in adapter for 18 mm tubes		96881
A-16 Adapter for 16 mm tubes		96882
CKG18 calibration kit for DEN-1, for 18mm glass tubes		96883
CKG16 calibration kit for DEN-1, for 16mm glass tubes		96884
Storage Tube with Desiccant	10 tubes	96719
McFarland Standard Set 0.5 - 1.0 - 2.0 - 3.0 -4.0	5 tubes	80405
0.5 McFarland	1 tube	80400
1 McFarland	1 tube	80401
2 McFarland	1 tube	80402

Ready to use culture media

DESCRIPTION	FORMAT	PACKAGING	REF.
MUELLER HINTON AGAR II for antimicrobial susceptibility testing of non-fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10031 10231
MUELLER HINTON AGAR II + SHEEP BLOOD 5% for antimicrobial susceptibility testing of fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10131 11231
MUELLER HINTON FASTIDIOUS AGAR (Horse blood 5% + 20 mg/L β -NAD) EUCAST recommended: for antimicrobial susceptibility testing of <i>Streptococcus</i> spp., <i>Haemophilus</i> spp., and some other fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10132 11132
MUELLER HINTON AGAR II + 2% NaCl for antimicrobial susceptibility testing of staphylococci with Oxacillin.	90 mm	20 plates	11206
CHROMATIC MH Chromogenic medium for the preliminary identification and susceptibility testing of bacteria directly from clinical and environmental specimens.	90 mm 140 mm	20 plates 10 plates	11618 10246
BRUCELLA BLOOD AGAR with HEMIN AND VITAMIN K1 for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	10245
SCHADLER K AGAR (SHEEP BLOOD 5%) for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	11065
HAEMOPHILUS TEST AGAR for <i>Haemophilus</i> spp. susceptibility testing.	90 mm	20 plates	10080
MIDDLEBROOK 7H11 AGAR for antimicrobial susceptibility testing of mycobacteria.	90 mm	20 plates	10416
R.P.M.I. AGAR (2% glucose + MOPS) for antifungal susceptibility testing.	90 mm 140 mm	20 plates 10 plates	11509 10233

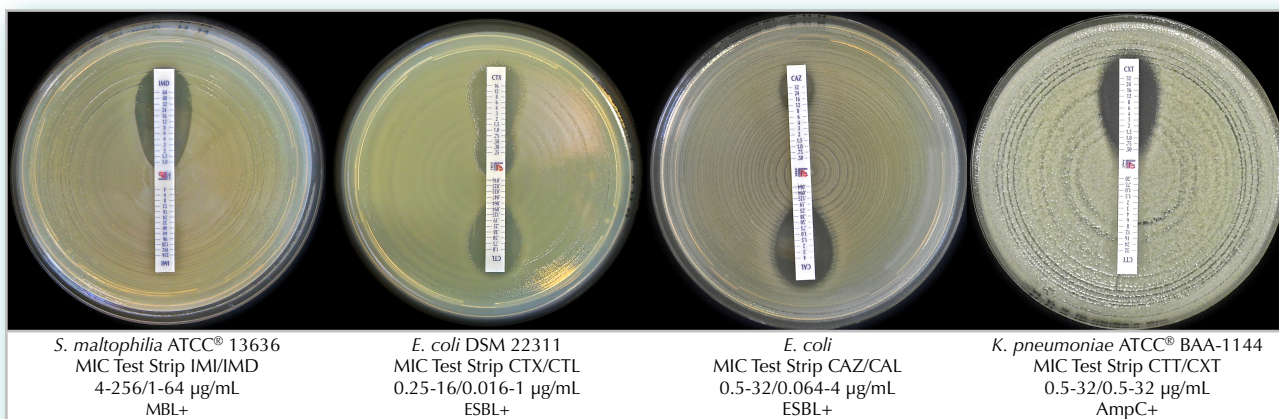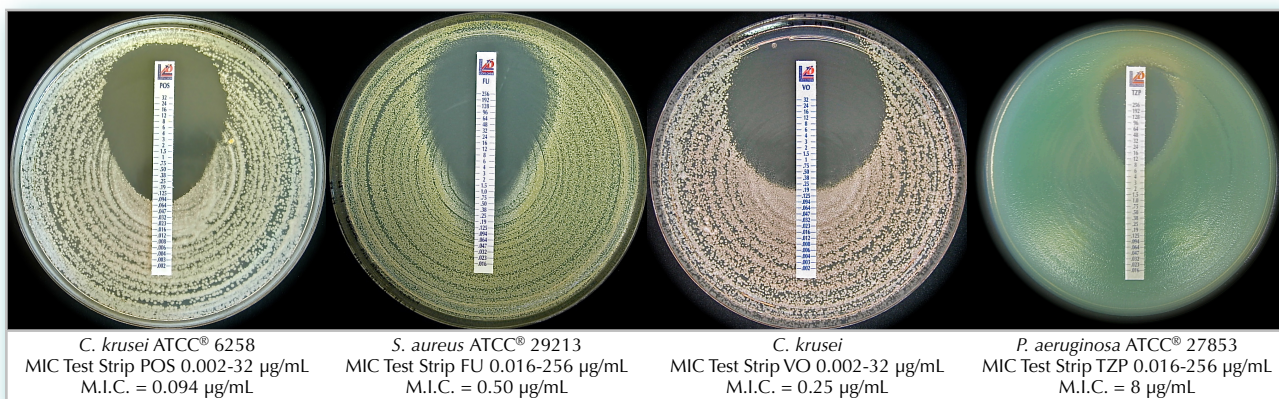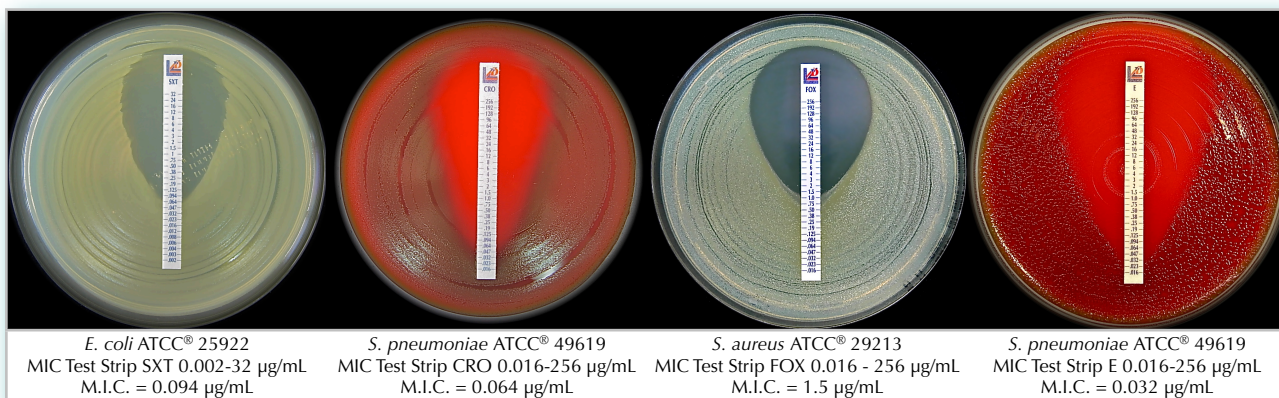

Contact your local dealer or Liofilchem headquarter for MIC Test Strip availability.



Argentina
MedicaTec S.R.L.
Tel +54 11 4554.4600
Fax +54 11 4555.0416
www.medica-tec.com



Denmark
Biotech-IgG A/S
Tel +45 3538 0500
Fax +45 2538 7322
info@biotech-igg.com



Malaysia
Premier Diagnostics Sdn Bhd
Tel +603-5191 3943
Fax +603-5191 3945
info-pd@premierdiagnostics.com.my



Singapore
Premier Diagnostics Pte Ltd
Tel +65 6220 3221
Fax +65 6220 9457
pdsingapore8@gmail.com



Australia
Integrated Sciences Pty Ltd
Tel +61 2 9417 7866
Fax +61 2 9417 5066
www.integratedsci.com.au



Estonia
Borsa Eesti OÜ
Tel +3725050555
Fax +3726556158
borsa@borsa.ee



Mexico
Abalat, S.A. de C.V.
Tel (01 55) 8000-1500
www.abalat.com.mx



Slovakia
Bio G spol. s r.o.
Tel +421245526381
Fax +421245526382
www.biog.sk



Austria
IVD Uhl-Michelin e.U.
Tel.: +43 (0) 69914616219
Fax: +43 (0) 463 265526 6222
werner.uhl-michelin@ivd.at



Finland
Immuno Diagnostic Oy
Tel +3583615370
Fax +35836822039
www.immunodiagnostic.fi



Netherlands
EWC Diagnostics B.V.
Tel +31 (0)6 15606050
carin.smale@ewcdiagnostics.com
www.ewcdiagnostics.com



Slovenia
Mediline d.o.o.
Tel +38618308040
Fax +38618308070
www.mediline.si



Belarus
Multi Lab Ltd.
Tel.: +375 17 309 17 20
Fax: +375 17 299 09 02
multilab@multilab.by



France
i2a
Tel +33467504800
Fax +33467170906
www.i2a.info



New Zealand
Point View Laboratories Ltd
Tel +64 9 265 2884
Fax +64 9 265 0776
info@pointviewlaboratories.com



South Africa
Davies Diagnostics (PTY) Ltd
Tel +2711777600
Fax +27117876917
www.daviesdiagnostics.co.za



Belgium
ELITech Group B.V.
Tel +32 (0)92820531
Fax +32 (0)92820532
c.notte@elitechgroup.com



Germany
bestbion dx GmbH
Tel +49 2234 987 95 0
Fax +49 2234 987 95 29
service@bestbion.com



Norway
Montebello Diagnostics AS
Tel 22 14 14 90
Fax 22 14 44 90
www.montebello.no



Spain
Werfen España, SAU
Tel 902203090
Fax 902223366
www.werfen.com



Brazil
Plast Labor
Tel (21) 25010888 (21) 25011905
(21) 3273 9105 . (21) 2599 8419
plabor@plastlabor.com.br



Greece
Varelas S.A.
Tel +302105281920
Fax +302105220926
microbiology@varelas.gr



Paraguay
GT Scientific S.A.
Tel 595 21 200134
Fax 595 21 221339
ventas@gtscientific.com.py



Sweden
Biotech-IgG AB
Tel +46 46 12 37 40
Fax +46 46 12 37 60
ak@biotech-igg.com



Bulgaria
Elta 90M
Tel +35929839649
Fax +35929832211
www.elta90.com



Hong Kong
Bio-Gene Technology Ltd.
Tel +85226466101
Fax +85226868806
info@bio-gene.com.hk



Poland
Argenta Spółka z ograniczoną odpowiedzialnością Sp.k.
tel. +48 618474637 fax +48 618483477
argenta@argenta.com.pl



Switzerland
Apteq AG
Tel: +41417400814
Fax: +41417403488
www.apteq.ch



Canada
Alere Canada
Tel (613) 271-1144
Fax (613) 271-1148
www.alere.ca



Hungary
Frank Diagnosztika Kft.
Tel 388-3114, 367-2430
Fax 368-5721
www.frank-diagn.hu



Portugal
Werfen Portugal, LDA
Tel +351 214161288
Fax +351 214176484
MIC Test Strip product range only



Thailand
Envimed Co., Ltd.
Tel 66-2-434-0391
Fax 66-2-435-5962
envimed@truemail.co.th



Chile
Medica Tec de Chile S.A.
Tel +56 (02) 4261830
fax +56 (02) 4261835
mrojas@medica-tec.cl



Ireland
Launch Diagnostics Ireland
Tel +3530719151122
Fax +3530719152277
desruddy@launchdiagnostics.ie



Russia
Agros-International, Zao
Tel +7 (812) 303-8234
Fax +7 (812) 303-8236
agros@agros-int.com



Turkey
Kemiteks Kimyevi Mad. Tic. Ltd. Sti.
Tel +90 216 449 58 70
Fax +90 216 449 58 72
www.kemitekskimya.com.tr



Colombia
Annar Diagnóstica
Tel 018000189999
Fax +57 1 7447979
ventas@annardx.com



Italy
Liofilchem
Tel +39 0858930745
Fax +39 0858930330
www.liofilchem.it



Saudi Arabia
Gamsco
Tel +966-3-8394085
Fax +966-3-8394084
drmalousef@yahoo.com



United Kingdom
Launch Diagnostics Ltd
Tel +44(0)1474874426
Fax +44(0)1474872388
enquiries@launchdiagnostics.com



Czech Republic
BioVendor
Tel +420549124123
Fax +420549211465
www.biovendor.cz



Lithuania
Biometrija
tel +37 052481824
fax +37 052400085
www.biometrija.lt



Serbia
Uni-Chem
Tel +381113282986
Fax +381113281205
www.uni-chem.rs



United States
Liofilchem, Inc.
Waltham, MA
tel 781-902-0312
www.liofilchem.net/en/contact.php



Liofilchem®

Via Scozia zona ind.le, 64026 Roseto degli Abruzzi (Te) Italy
Tel. +39 0858930745 Fax +39 0858930330 www.liofilchem.net



In USA, available for products noted as "FDA Cleared" in the product list.

ref. 6553003
Rev.35 / 21.12.2017

lioilchem@lioilchem.net