S F Rapid Test

Rapid test for differentiation of enterococci from other *Streptococcus* spp.

INTENDED PURPOSE

Rapid test for the identification of group D streptococci from clinical specimens. This test is intended as an aid in the diagnosis, requiring further tests to complete the diagnostic results.

DESCRIPTION

S F Rapid Test is used for the differentiation of enterococci from other *Streptococcus* species, including group D non-enterococci.

KIT CONTENT

- 30 tubes of S F Rapid Test.

METHOD PRINCIPLE

Streptococci are Gram-positive, nonmotile, nonspore-forming, catalase-negative cocci that occur in pairs or chains. Older cultures may lose their Gram-positive character. Most streptococci are facultative anaerobes, and some are obligate (strict) anaerobes. Most require enriched media (blood agar). Streptococci are classified based on colony morphology, hemolysis, biochemical reactions, and (most definitively) serologic specificity (Lancefield serotyping). Lancefield group D streptococci includes *Streptococcus bovis* group and Enterococcus species. Enterococci ferment dextrose and grow in the presence of the inhibitor sodium azide.

S F Rapid Test consists of a dehydrated culture media (see the medium composition under REAGENTS). Enzymatic digest of casein provides amino acids, nitrogen, carbon, vitamins and minerals for organisms growth. Dextrose is the fermentable carbohydrate. Sodium azide, colistin, nalidixic acid, kanamycin and crystal violet are selective agents inhibiting most bacteria with the exception of *enterococci*. Bromcresol purple is the pH indicator.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: incubator, inoculating loop, pipettes, culture media, physiological solution (0.85% saline), test tubes, quality control organisms.

REAGENTS

- A tube of S F Rapid Test contains:

TYPICAL FORMULA*	(g/litre)
Enzymatic Digest of Casein	100.00
Dextrose	50.00
Sodium Azide	0.10
Colistin Sulphate	0.10
Nalidixic Acid	0.15
Kanamycin Sulphate	0.20
Crystal Violet	0.43
Bromcresol Purple	0.32
Final pH 7.0 ± 0.2	

*Adjusted and/or supplemented as required to meet performance specifications.

SPECIMEN

Collect specimens in sterile containers or with sterile swabs and transport to the laboratory. Process each specimen using procedures appropriate for that sample. This product is recommended for use only with pure cultures.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

1. Take the number of S F Rapid Test tubes needed from the fridge and allow them to reach room temperature.
2. Add 0.3 ml of sterile physiological solution to each tube.
3. Inoculate heavily with the test organism from a fresh overnight pure culture.
4. Incubate at $35 \pm 2^\circ\text{C}$ for up to 24 hours.
5. Examine tubes at intervals of 2, 12, 18 and 24 hours for broth colour change.

Note: Positive and negative controls should be run simultaneously with the organism to be tested (see QUALITY CONTROL).

INTERPRETING RESULTS

A positive reaction for the presence of group D enterococci is indicated by turbidity and a colour change of the medium from purple to yellow. A negative reaction is indicated by absence of growth and a purple colour.

Streptococci yielding positive reactions:

E. faecalis, *E. faecium*.

Streptococci yielding negative reactions:

S. bovis, *S. equinus*, *S. mitis*, *S. salivarius*, *Streptococcus* species other than group D.

STORAGE

2-8°C in its original packaging. Keep away from sources of heat and avoid excessive changes of temperature. Use until the expiry date indicated on the label. Eliminate without using if there are signs of deterioration.

SHELF LIFE

1 year.

QUALITY CONTROL

Appearance of S F Rapid Test: Yellowish powder; Deep purple, clear broth, once reconstituted.

Control strain		Incubation	Characteristic reactions
<i>Escherichia coli</i>	ATCC® 25922	up to 24 h/ $35 \pm 2^\circ\text{C}$	Negative reaction: No colour change (broth remains purple)
<i>Staphylococcus aureus</i>	ATCC® 25923		
<i>Streptococcus pyogenes</i>	ATCC® 19615		
<i>Enterococcus faecalis</i>	ATCC® 19433		Positive reaction: Colour change to yellow

PERFORMANCE CHARACTERISTICS

Performance testing of S F Rapid Test was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

Some enterococci strains may take 48 hours to cause a color change of the medium while others grow with no color change even after 72 hours.

The test will respond only to the presence of the group D streptococci and will not detect other bacteria.

WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.

- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document.

ORDER INFORMATION

Product	Packaging	Ref.
S F Rapid Test	30 tests	88020

Revision History

Revision	Release Date	Change Summary
1	2024-09-16	Added notice to report any malfunction, defect or incident.
0	2023-05-15	Updated layout and content in compliance with IVDR 2017/746, version reset to revision 0

In case of malfunctions or defects, contact immediately Liofilchem (*) or the local representative.

In case of incident associated with the device, notify immediately Liofilchem (*) or its local representative and the National Competent Authority.

*Please login to <https://www.liofilchemstore.it/login.php> (user ID and password required) and click on Complaint.

This IFU document and the SDS are available from the online Support Center:

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References / Riferimenti

1. Public Health England (2021) UK SMI ID 4 issue 4: Identification of Streptococcus species, Enterococcus species and morphologically similar organisms.
2. Facklam, R.R. Recognition of Group D Streptococcal species of Human origin by Biochemical and physiological tests. Appl. Microbiol., 23:1131 – 1139, 1972.
3. Hajna, A.A. and Perry, C.A. Comparative Study of presumptive and confirmative media for bacteria of the Coliform Group and for Faecal Streptococci. Am J. Public Health, 33:550 – 556, 1943.

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