MIF Color Kit

Kit for the detection of fecal parasites in feces

INTENDED PURPOSE

Staining and fixative solution for the detection of eggs and faecal parasite.

DESCRIPTION

MIF Color Kit allows for the quick and easy detection of parasites in feces, especially protozoa (amoeba).

This method simultaneously corrects and stains the sample providing an optimal image for visualization of the nuclear structures necessary for the identification of some types of protozoa. The vegetative and cystic forms of parasites have different staining affinities. MIF Color Kit allows for quality staining of parasites and provides concentration of parasitic elements on the sediment surface, but due to quality fixation and storage of specimens for an extended period.

KIT CONTENT

- 50 Bottles of A Solution
- 1 Bottles of B Solution

METHOD PRINCIPLE

Two-reagent merthiolate-iodine-formalin kit for fixing and staining fecal parasites (especially protozoa, cysts, helminthic eggs, and larvae). The stool sample is fixed with formalin and stained with two coloring agents: Iodine and Eosin.

MATERIALS REQUIRED BUT NOT PROVIDED

Standard microbiological supplies and equipment such as: inoculating loop, pipettes, microscope, cover glass.

REAGENTS

Formula	(g/l)
<u>A SOLUTION: MIF</u>	
Merthiolate	0.465
Formalin 37%	12.56
Eosin	0.037
Glycerol	9.3
<u>B SOLUTION: LUGOL</u>	
Iodine	100
Potassium Iodide	50

SPECIMENS

The patient must be suspending any therapy with antacid, antidiarrhoeic, antimicrobial and laxative drug.

The faecal samples must be collected in clean, large container or on clean paper sheet; do not utilize faeces from toilettes or mixing with urine.

Send the sample to laboratory within 1 hour to emission.

Refer to specific guidelines for more detailed information.

TEST PROCEDURE

1. Take 1 ml of B solution and transfer it into the A solution container.
2. Take with apposite loop a little quantity of faeces and crush it against the internal side of flask.
3. Close up the lid and mix until the homogeneous suspension of sample.
4. Leave to replace for 1 hour.
5. Take with a pipette a little quantity of sample and to put down a drop on microscope slide.
6. Cover the microscope slide and examine at microscope.

INTERPRETING RESULTS

Vegetative and cystic forms are stained a pink brown. Eggs and larvae of parasites are visible but not stained.

STORAGE

Storage at 10-25°C away from light, until the expiry date on the label or until signs of deterioration or contamination are evident.

SHELF LIFE

2 years

PERFORMANCE CHARACTERISTICS

Performance testing of MIF Color Kit was carried out using the QC strains listed above. The results obtained met the established criteria.

LIMITATIONS

- Examination of fresh stools is complementary to concentration methods and must always be carried out on stools at +37°C in order to avoid the destruction of certain vegetative forms;
- Under no circumstances exceed the sedimentation time. Indeed, some large parasite eggs could sediment and thus lead to a false negative.
- MIF Color Kit is recommended in all cases regardless of the stool consistency at the start.

WARNING AND PRECAUTIONS

- 1) **For *in vitro* diagnostic use (IVD).**
- 2) **For laboratory professional use only.**
- 3) Operators must be trained and have certain experience. Please read the instructions carefully before using the product. Reliability of assay results cannot be guaranteed if there are any deviations from the instructions in this document.
- 4) Consult the Safety Data Sheet (SDS) for information regarding hazards and safe handling practices.
- 5) Do not use if the product or packaging appears to be damaged.
- 6) Follow standard precautions. All patient specimens should be considered potentially infectious and handled accordingly.
- 7) Handle all specimens as if infectious using safe laboratory procedures. Dispose of hazardous or biologically contaminated materials according to the practices of your institution.
- 8) Avoid cross-contamination of samples by using disposable tips and changing them after each sample.
- 9) Do not mix reagents of different batches. Please use the product within the validity period.
- 10) Do not eat, drink, smoke, apply cosmetics or handle contact lenses in areas where reagents and human specimens are handled.
- 11) Results should be interpreted by a trained professional in conjunction with the patient's history and clinical signs and symptoms, and epidemiological risk factors.
- 12) Ensure laboratory equipment is calibrated and maintained in accordance with the laboratory's procedure.
- 13) When test results are transmitted from the laboratory to an informatics centre, attention has to be done to avoid erroneous data transfer.

DISPOSAL OF WASTE

Disposal of waste must be carried out according to national and local regulations in force.

BIBLIOGRAPHY

See the references at the end of this document.

TABLE OF SYMBOLS

See the table of symbols at the end of this document

ORDER INFORMATION

Product	Packaging	Ref.
MIF Color Kit	50 x 12 ml	80275

Revision History

Revision	Release Date	Change Summary
1	2024-09-16	Added notice to report any malfunction, defect or incident.
0	2023-05-15	Updated layout and content in compliance with IVDR 2017/746, version reset to revision 0

In case of malfunctions or defects, contact immediately Liofilchem (*) or the local representative.

In case of incident associated with the device, notify immediately Liofilchem (*) or its local representative and the National Competent Authority.

*Please login to <https://www.liofilchemstore.it/login.php> (user ID and password required) and click on Complaint.

This IFU document and the SDS are available from the online Support Center:

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