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Rev.1 / 14.05.2012

Rapid agglutination test for direct identification of *Clostridium difficile*

DESCRIPTION

CLOSTRIDIUM DIFFICILE LATEX KIT is a rapid latex agglutination test on card for direct identification of *Clostridium difficile*, both from culture media in plate, and from enrichment broths.

PACKAGE CONTENT

- 1 LATEX REAGENT (2.5 mL): dropper bottle containing latex particles coated with antibodies raised against *Clostridium difficile*.
- 1 POSITIVE CONTROL (1 mL): dropper bottle containing positive control composed of inactivated antigens of *Clostridium difficile*
- 10 x 6 wells disposable cards.
- 2 x 25 disposable mixing sticks.

NECESSARY ITEMS NOT CONTAINED IN THE PACKAGE

- Disposable pipettes
- Disposable loops
- Physiological solution

METHOD PRINCIPLE

Latex particles in CLOSTRIDIUM DIFFICILE LATEX KIT are individually coated with antibodies against *Clostridium difficile*. Mixed with a suspension of *Clostridium difficile*, latex particles aggregate to give visible agglutination.

TEST PROCEDURE

- Allow the reagents and positive control to reach room temperature.
- METHOD FROM CULTURE MEDIUM IN PLATE: dispense one drop of physiological solution on an agglutination card. Using a disposable loop remove one well-isolated colony from an agar plate and emulsify it in the drop of physiological solution so to obtain a heavy and homogeneous suspension. Suspensions must only be made from colonies with morphologies resembling *Clostridium difficile*.
- METHOD FROM ENRICHMENT BROTHS: using a disposable pipette, take one drop from the suspension in enrichment broth and drop it on an agglutination card.
- Dispense one drop of POSITIVE CONTROL on a well of the agglutination card.
- Dispense one drop of CLOSTRIDIUM DIFFICILE LATEX REAGENT on the agglutination card next to the bacterial suspension, and one drop on the well with the positive control. Do not allow the dropper to touch bacterial suspension.
- Mix the latex reagent and the bacterial suspension with a clean mixing stick and rock the slide gently two or three times. Excessive rocking of the slide is not necessary. Examine for agglutination within a maximum of 2 minutes.

INTERPRETATION OF RESULTS

Agglutination within 2 minutes is a positive result and indicates the presence of *Clostridium difficile* in the sample. Absence of agglutination indicates that *Clostridium difficile* is not present in the test culture. Reactions that are "curdy" or "stringy" may not be true agglutination.

LIMITATION OF USE

- Results should be interpreted by clinician in the context of all available clinical and laboratory information. The isolation of *C. difficile* does not constitute a diagnosis of pseudomembranous colitis or antibiotic-associated diarrhoea.
- Identification of *C. difficile* using CLOSTRIDIUM DIFFICILE LATEX KIT should be performed on cultures grown on selective media as this increases the isolation rate or from a purity plate such as Clostridium Agar (ref. 11060).
- Culture-derived suspensions which auto-agglutinate cannot be tested by CLOSTRIDIUM DIFFICILE LATEX KIT. Alternative methods should be used.

PRECAUTION

CLOSTRIDIUM DIFFICILE LATEX KIT is designed for *in vitro* diagnostic use and must be used only by properly trained operators, using approved aseptic and safety methods for handling pathogenic agents.

STORAGE

Store CLOSTRIDIUM DIFFICILE LATEX KIT at 2-8 °C in the original packaging. Keep away from sources of heat and avoid excessive changes in temperature. In such conditions CLOSTRIDIUM DIFFICILE LATEX KIT will remain valid until the expiry date indicated on the label. Do not use beyond that date. Eliminate without using if there are signs of deterioration.

DISPOSAL OF USED MATERIAL

After use CLOSTRIDIUM DIFFICILE LATEX KIT and material that has come into contact with the sample must be decontaminated and disposed of in accordance with the techniques used in the laboratory for decontamination and disposal of potentially infected material

PRESENTATION

Product	REF	Σ
CLOSTRIDIUM DIFFICILE LATEX KIT	96144	50 tests

TABLE OF SYMBOLS

IVD <i>In vitro</i> diagnostic medical device	 Do not reuse	 Manufacturer	 Content sufficient for <n> tests	 Temperature limits
REF Catalogue number	 Fragile, handle with care	 Use by	 Attention, see instructions for use	LOT Batch code

LIOFILCHEM® S.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@lioilchem.net

Rev.1 / 14.05.2012