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E-Mail: [liofilchem@liofilchem.net](mailto:liofilchem@liofilchem.net)

F12411  
Rev.1 / 22.07.2009

## Biological indicators of steam sterilization processes containing *Geobacillus stearothermophilus* (ATCC 7953) spores inoculated on special filter paper strips

### DESCRIPTION

USP (United States Pharmacopoeia), EP (European Pharmacopoeia) and DAB (Deutsches Arzneibuch) standards, recommend to use bioindicators during steam sterilization processes of pharmaceuticals, drugs in vials, culture media, medical instruments and more. Biological indicators **STRIP CONTROL GST E8/E6/E5/E4/E3** are special filter paper strips, inoculated with *Geobacillus stearothermophilus* (ATCC 7953) spores in predefined concentrations and contained in a special envelope. These bioindicators are used for regular control of steam sterilization cycles (for ie. 15 minutes at 121 °C) and control of any steam autoclave functionality. Biological indicators **STRIP CONTROL GST E8/E6/E5/E4/E3** are produced under strictly controlled conditions in order to satisfy the requirements indicated in the USP current edition and in accordance with ISO 11138 and EN 866 standards.

### PRINCIPLE

STRIP CONTROL GST E8/E6/E5/E4/E3 contain *Geobacillus stearothermophilus* (ATCC 7953) spores in predefined concentrations: E8=1-5x10<sup>8</sup>CFU/strip; E6=1-5x10<sup>6</sup>CFU/strip; E5=1-5x10<sup>5</sup>CFU/strip; E4=1-5x10<sup>4</sup>CFU/strip; E3=1-5x10<sup>3</sup>CFU/strip. Spores in the strips are completely killed off after 15 minutes of sterilization at 121 °C and therefore during the following incubation in the Steri-Test Medium tubes, included in the package, they are not able to grow and to modify medium's colour. At a lower temperature or at a shorter sterilization time, spores partially survive and therefore during the incubation are able to grow and to modify medium's colour.

### INSTRUCTIONS FOR USE

- Take one or more strips from the kit leaving them in their original envelope.
- Put the envelopes with the strips on the bottom, in the centre, inner sides and on the critical points of the autoclave.
- For autoclaves with capacity up to 250 litres put two envelopes for each selected point of the autoclave. For autoclaves with capacity higher than 250 litres put six or more envelopes in each selected point.
- Remove the original envelopes after sterilization cycle and open them aseptically with sterile scissors or by tearing the edges.
- Transfer aseptically each strip from its envelope to a Steri-Test Medium tube included in the package.
- Incubate tubes containing strips at 55-60 °C (131-140 °F) for 7 days or for a shorter time validated by user.
- Incubate, at the same conditions of time and temperature, a strip contained in the envelope not submitted to the sterilization cycle, belonging to the same batch, as spore growth control (positive control).
- Examine tube medium's colour and interpret results as per **EVALUATION TABLE**: a change of medium's colour from violet/clear to yellow/turbid indicates microbial growth and therefore an unsuccessful sterilization. On the contrary, the persistence of the medium's initial colour (violet/clear) indicates absence of microbial growth and therefore a successful sterilization.

### EVALUATION

*Geobacillus stearothermophilus* spores are killed off if the sterilization cycle has been efficient: in this case the medium's colour remains violet/clear even after incubation at 55-60 °C (131-140 °F) for the selected time.

If the sterilization cycle has not been efficient, spores partially survive and the tube's content turns yellow/turbid after incubation at 55-60 °C (131-140 °F) for the selected time.

The tube inoculated with the strip contained in the envelope not submitted to the sterilization cycle and used as spore growth control, has to turn yellow/turbid after incubation. On the contrary, the test must be repeated after having investigated the causes of the negative result.

EVALUATION TABLE		
MEDIUM'S COLOUR	SPORE	STERILIZATION
Violet / Clear	Killed off	Successful
Yellow / Turbid	Vital	Unsuccessful

### TREATMENT OF STERITEST MEDIUM TEST TUBES AFTER USE

After use, sterilize the positive tubes (yellow/turbid) in autoclave at 121 °C for at least 30 minutes and eliminate them in accordance with the procedures of the laboratory.

### STORAGE

Store the product at 2-8 °C: in these conditions it maintains its validity until the expiry date indicated on the label.

### BIBLIOGRAPHY

- United States Pharmacopoeia latest edition.
- Deutsches Arzneibuch latest edition.
- European Pharmacopoeia latest edition.
- ISO 11138 and EN 866 latest edition.

### PRESENTATION

PRODUCT	CODE	PACKAGING	Spores: CFU / strip	D <sub>121°C</sub>
STRIP CONTROL GST E8	91060	20 envelopes+20 Steri-Test Medium tubes	1-5x10 <sup>8</sup>	1,5-3,0 minutes
STRIP CONTROL GST E6	91055	20 envelopes+20 Steri-Test Medium tubes	1-5x10 <sup>6</sup>	1,5-3,0 minutes
STRIP CONTROL GST E5	91056	20 envelopes+20 Steri-Test Medium tubes	1-5x10 <sup>5</sup>	1,5-3,0 minutes
STRIP CONTROL GST E4	91057	20 envelopes+20 Steri-Test Medium tubes	1-5x10 <sup>4</sup>	1,5-3,0 minutes
STRIP CONTROL GST E3	91058	20 envelopes+20 Steri-Test Medium tubes	1-5x10 <sup>3</sup>	1,5-3,0 minutes

### TABLE OF SYMBOLS

 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Caution, consult accompanying documents
 Use by	 Batch code	 Do not reuse

**LIOFILCHEM Bacteriology Products**

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745

Fax +390858930330

Website: [www.liofilchem.net](http://www.liofilchem.net)

E-Mail: [liofilchem@liofilchem.net](mailto:liofilchem@liofilchem.net)

F12411  
Rev.1 / 22.07.2009